

Dolomite® Alpha Basic / Dolomite® Alpha Advanced

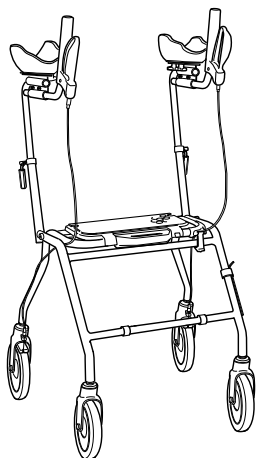Alpha Basic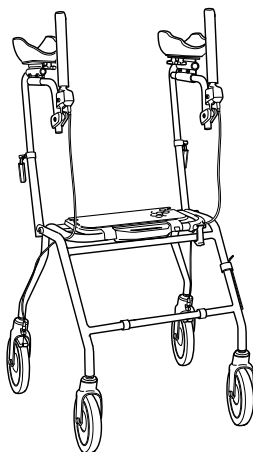

Alpha Advanced

en	Walking table User Manual	3
da	Ganghjælpemiddel Brugsanvisning	17
de	Gehhilfe Gebrauchsanweisung	31
fi	Kävelypöytä Käyttöohje	47
fr	Tablette de déambulation Manuel d'utilisation	61
it	Deambulatore Manuale d'uso	77
nl	Looptafel Gebruiksaanwijzing	93
no	Gåbord Bruksanvisning	109
sv	Gåbord Bruksanvisning	123

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

© 2016 Invacare® Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

1 General	4
1.1 About this manual	4
1.2 Intended use	4
1.3 Service Life	4
1.4 Scope of delivery	5
1.5 Warranty	5
2 Safety	6
2.1 General Warnings	6
2.2 Labels and symbols on the product	6
3 Usage	7
3.1 Before use	7
3.2 Assembling the walking aid	7
3.3 Folding and unfolding the walking aid	8
3.4 Adjusting the arm support	8
3.5 Adjusting the brake handle (Alpha Advanced)	10
3.6 Braking	10
3.7 Walking	11
3.8 Sitting down on a chair	11
3.9 Rising to a standing position	12
3.10 Sitting on the walking aid	12
4 Maintenance and care	13
4.1 Maintenance	13
4.2 Cleaning	13
5 After Use	14
5.1 Disposal	14

5.2 Re-use	14
6 Technical data	15
6.1 Dimensions and weights	15
6.2 Material	16

I General

I.1 About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

I.2 Intended use

The walking aid is intended for support and improvement of a limited walking ability. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others.

The walking aid is intended to support a person during walking and may also serve as a resting seat. Furthermore the walking aid is intended to carry a load, either in a specially designed basket or on a tray.

The walking aid is not intended for use by persons with severe equilibrium disorders or functional perceptual disorders.


Permissible operating conditions and locations

The walking aid is only intended for use indoors, in living or public areas, or outdoors. The walking aid is not to be used under water in e.g. pools or open-air bath, neither in hot areas as e.g. saunas.

I.3 Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

1.4 Scope of delivery

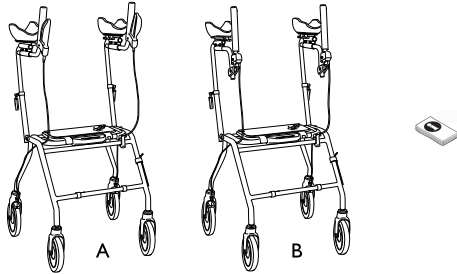A: Alpha Basic

B: Alpha Advanced

- Walking aid, partly pre-assembled.
- User manual

For assembling see section 3.2 Assembling the walking aid, page 7 .

Accessories, actually assembled to your walking aid, are depending on the configuration and might differ per region.

1.5 Warranty

We supply a two year warranty from the date of delivery. Deviations may occur due to local agreements. Damage caused by normal wear to e.g. wheels, handles and brakes or damage caused by abnormal use (e.g. exceeded max. user weight) is not covered by the warranty. Maintenance and operating instructions must be followed for the warranty to be valid.

2 Safety

2.1 General Warnings

WARNING!

Risk of falling

- Exercise caution when walking on sloping / slippery ground and on uneven surfaces.
- Do not use the walking aid as a wheelchair.
- Be careful when walking rearwards over obstacles.
- Never sit and push with your feet.
- Do not go up or down stairs with the walking aid.
- Do not use the walking aid on escalators.
- Do not under any circumstances stand on the walking aid.
- Be careful when using long skirts, the clothes might get stuck in the wheels.
- Loads must only be transported in the basket or on the tray. Be careful when there is a heavy load in the basket (for the max. load in the basket refer to chapter “Technical data”).

WARNING!

Risk of injury

- Do not let children play with the walking aid.
- If the push handles, underarm pads or the seat are exposed to extreme temperature (above 41°C) there might be risk for first degree burns.

WARNING!

Incorrect use could lead to hazardous situations

- Invacare accepts no responsibility for use, change or assembly of the product, other than stated in this user manual.


IMPORTANT!

Risk of corrosion

- The walking table must not be stored in wet or humid environments.

2.2 Labels and symbols on the product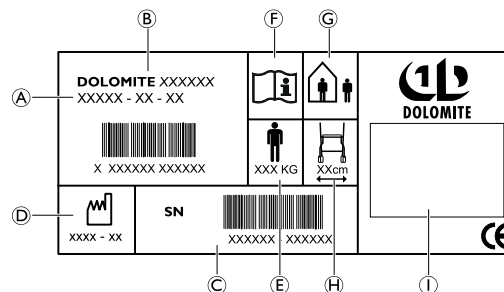

The label contains all important technical information regarding your walking aid.

- | | |
|------------------------------|--------------------------------------|
| A Article number | F Refer to user manual |
| B Product name | G In- and outdoor walking aid |
| C Serial number | H Max. width of walking aid |
| D Date of manufacture | I Manufacturer's name. |
| E Max. user weight | |


3 Usage

3.1 Before use


We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.

3.2 Assembling the walking aid

Alpha Basic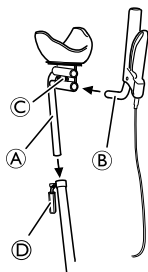


Alpha Advanced

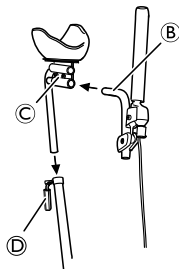1. Place the arm support tube ① in the frame tube and secure with lever ④.
2. Place the handle bar ② in the arm support tube and secure with the screws ③.
3. Repeat the procedure on the other side.

The arm support tube and the handle bar can be placed in two different positions depending on the required height.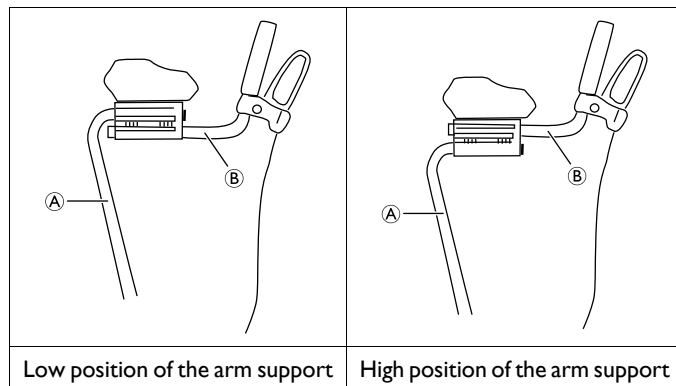

Low position:


1. Place the arm support tube ① in the upper attachment position.
2. Place the handle bar ② in the lower attachment position.

High position:

1. Place the arm support tube ① in the lower attachment position.
2. Place the handle bar ② in the upper attachment position.

3.3 Folding and unfolding the walking aid

Unfolding to operating position

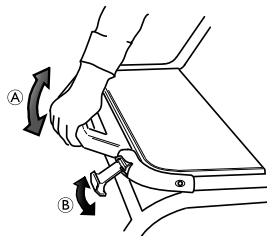1. Unfold the walking aid by pressing down the seating tube using the handle (A).


WARNING!

- Before using the product, always check that the safety latch (B) is in locked position indicated by a clear audible “click”.

Flip up the seat plate

The seat can be folded up to provide a larger walking space, and in order to increase accessibility.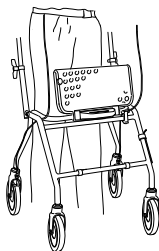

Folding the walking aid


1. Follow the action above in reverse.

3.4 Adjusting the arm support


WARNING!


- All adjustments must be carried out by your specialist dealer.
- Before usage, always make sure that the levers (A) are pointing downwards and correctly tightened after height or angle adjustment.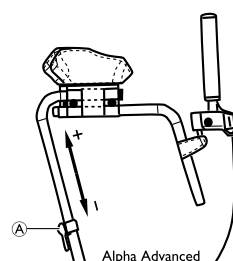

Alpha Advanced

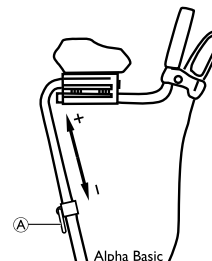Alpha Basic

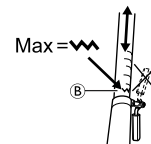Adjusting the height

1. Untighten the lever (A).
2. Adjust the arm support to a suitable height.
3. Tighten the lever.

WARNING!

Risk of falling


- Do not set the arm support above the highest position, indicated by the marking (B) on the tubes.

Adjusting the angle

1. Untighten the lever (A).
2. Adjust the angle of the handles to a suitable position.

Make sure that the handles are always in the allowed adjustment area (maximum 20° inwards).


Do not turn the handles outwards.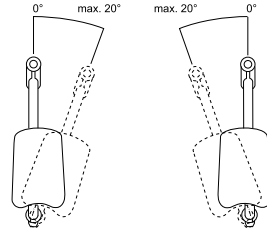


3. Tighten the lever.

Adjusting depth and inclination angle

Alpha Basic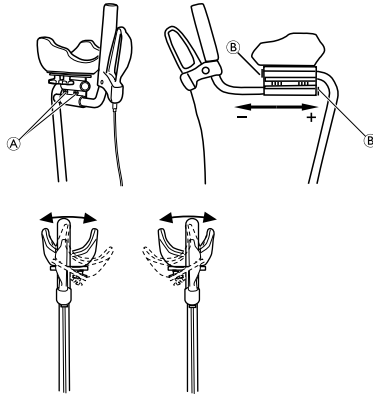


Alpha Advanced

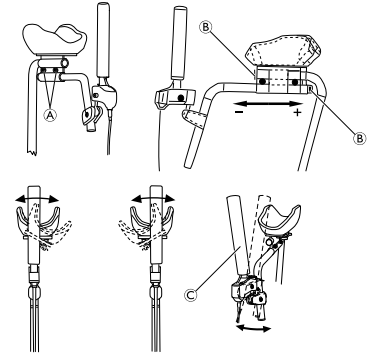1. Loosen the two screws (A) with the supplied Allen key.
2. Adjust the depth of the arm support to the desired position. Make sure that the black end plugs (B) are always visible.
3. Adjust the inclination angle of the arm support cup to the desired position.
4. Adjust the angle of the brake handle to the desired position.
5. Tighten the screws carefully.


Rotating the arm support cups inwards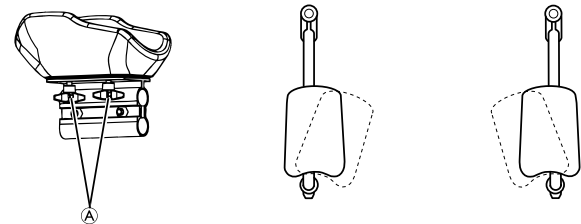

1. Loosen the two knobs ①.
2. Adjust the two arm support cups to appropriate width/angle.
3. Tighten the two knobs ①.

3.5 Adjusting the brake handle (Alpha Advanced)

The brake handle can be additionally adjusted for optional hand location and function direction.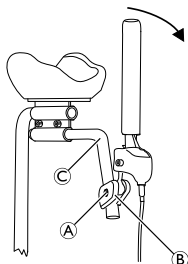

1. Loosen the screw ① with the supplied Allen key.
2. Adjust the fastening device ② in height and sideways to the desired position.
3. Rotate the brake by means of the ball joint to the desired position.
4. Retighten the screw ①.


The brake can be set at a greater angle by twisting the pipe ③ in connection with the depth adjustment, ③ 3.4 Adjusting the arm support, page 8.


3.6 Braking

Braking during usage

Alpha Basic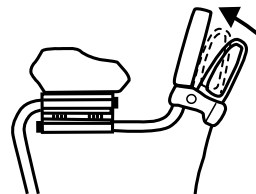

Alpha Advance

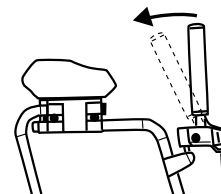1. Pull both brake handles (Alpha Basic) or brake levers (Alpha Advance) towards your body.


Applying the parking brakes


IMPORTANT!

- In parking position, always ensure that both brakes are applied.

Alpha Basic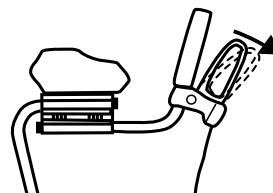


Alpha Advance

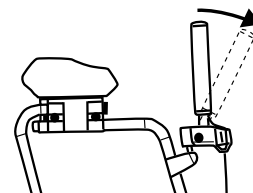1. Push both brake handles (Alpha Basic) or brake levers (Alpha Advance) away from your body.
A clear audible "Click" indicates that the brake is in the right position.
2. To release the brakes, pull the brake handles (Alpha Basic) or brake levers (Alpha Advance) backwards.

Adjusting the brake effect

For desirable braking strength, adjust the brake wire.

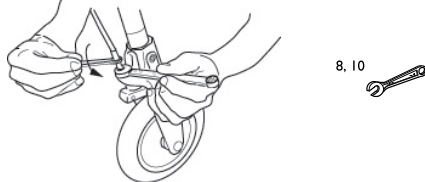Suitable tools: 8 mm and 10 mm spanner (not included).

1. Loosen the locking nut and turn the hexagon nipple counter clockwise in order to increase the braking effect. The distance between the brake clamp and the wheel should be 2 mm.
2. Tighten the locking nut again to fixate the new adjustment.
3. Check the braking effect by engaging the parking brake and check that the wheel can't rotate.

WARNING!

– Make sure to tighten the lock nuts after adjustment.


3.7 Walking

WARNING!


Risk of falling

– Be careful if the wheels stop against an obstacle.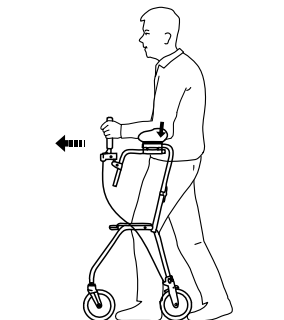

1. Place the walking aid a bit in front of you and take a step towards it whilst using the hand grips for support.

3.8 Sitting down on a chair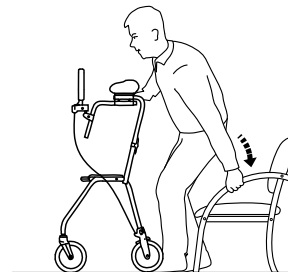

1. Stand in front of the chair with the walking aid, the back of the legs should touch the chair.
2. Place the hands on the arms of the chair, lean forward slightly and sit down carefully.


3.9 Rising to a standing position

WARNING!

Risk of tipping

- Never use the walking aid to pull yourself up from a sitting position.
- The parking brakes must be in braked position when the walking aid is used as a support when getting up, from a chair for example.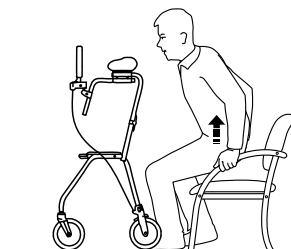

1. Place the walking aid in front of you, use the armrests of the chair as support and rise carefully to a standing position.
2. Once steady in a standing position, transfer hands to the hand grips of the walking aid.

3.10 Sitting on the walking aid

WARNING!

Risk of falling

Use the walking aid for stationary seating only. Both brakes must be in parking position when the walking aid is used as a seat.

- Never sit and push with your feet.
- Never sit on the walking aid on sloping ground.

4 Maintenance and care

4.1 Maintenance

Maintenance should be done at least once a year.

To ensure good performance and safety you should regularly check that the following is carried out:

- Ensure that screws and adjustments are properly tightened.
- Never use a defective walking aid. Do not attempt repairs yourself!
- If hand grips are loose do not use the walking aid. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Verify brake operation. If brake adjustment is required, see chapter 3 Usage, page 7 in this user manual.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

4.2 Cleaning

The product can be cleaned and disinfected using commercially available detergents.

IMPORTANT

- Cleaning must be done at least once a year.
- Do not use an abrasive detergent cleaner.
- Do not use a steam cleaner.
- Never use acids, alkalies or solvents.

- The product can be washed in a washing machine or by hand.
- The washing and drying temperature may not be higher than 60° C.

- High-pressure cleaner or ordinary jet of water may be used. However, do not aim the jet towards bearings, draining holes or labels.
- Clean with damp cloth and mild detergents.
- Clean the wheels with a brush (no wire brush!).
- After washing, wipe the product dry.

Disinfection

Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

5 After Use

5.1 Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way.

The individual parts can be separated and handled according to their material.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Re-use

This product is suitable for re-use after the following has been performed:

- Inspection
- Cleaning

For details, see chapter 4 Maintenance and care, page 13.

WARNING!

Risk of injury


- Always inspect the product according to the service instructions before transferring the product to a different user.
- If any damage is found do not use the product.

The user manual belongs to the product. Make sure that the user manual will be handed over with the product.

6 Technical data

6.1 Dimensions and weights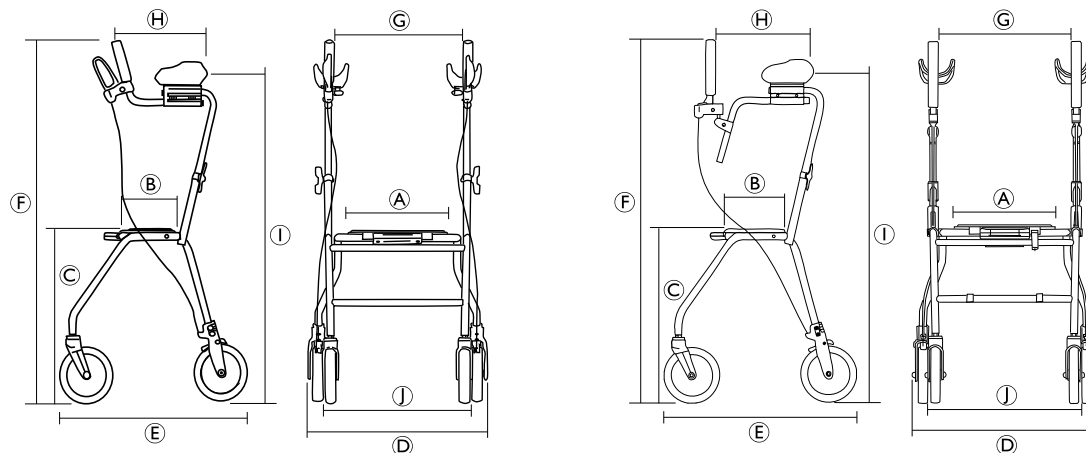

	Alpha Basic	Alpha Advanced
Seat width A	360 mm	
Seat depth B	180 mm	
Seat height C	620 mm	
Total width D	650 mm	
Total depth E	700 mm	
Total height (Handle height) F	1075 – 1310 mm	1165 – 1400 mm
Distance between handles G	430 mm	

	Alpha Basic	Alpha Advanced
Distance between handle and lower arm support ④	290 – 330 mm	310 – 360 mm
Arm support height ①	950 – 1220 mm	
Distance between rear wheels ①	510 mm	
Front/rear wheel ¹⁾ diameter	200 mm	

¹⁾ Front wheels are steering wheels. Rear wheels are provided with a brake system for dosed breaking and for locking the wheel.

	Alpha Basic	Alpha Advanced
Total weight ²⁾	11.3 kg	12.2 kg
Max. user weight ³⁾	125 kg	

²⁾ Total weight excluding equipment.

³⁾ This weight limitation includes the weight of the user whether standing or sitting and the content of the basket.

6.2 Material

Alpha Basic / Alpha Advanced	
Frame	Polyester (PES) powder coated steel
Handle bars	Aluminium
Handgrips	Closed cellular synthetic rubber
Wheels	Polypropylene (PP) with treads of Thermoplastic Elastomers (TPE)
Plastic parts	Material according to marking Polyamide (PA), Polypropylene (PP), Polyethylene (PE)
Screws	Steel, rust protected (zinc plated)

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Invacare Poirier SAS
Invacare Rea AB