

MAINTENANCE – SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Use a soft, damp cloth to clean away dust and dirt. Use soap and water to remove stubborn stains. Dry with a soft cloth. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.
- Check that nuts and bolts are properly tightened.
- If your walking support is not functioning perfectly - contact your dealer immediately.
- Never use a defective walking support. Do not attempt repairs yourself!
- The life cycle of the product is 5 years.


NB. Only to be used as a walking support. User's max. weight: P402, P408 100 kg and P421B 120 kg. **INVACARE** will not accept responsibility for adaptations, faulty repairs etc. that are outside its control.

INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKAGING:


When the product can no longer be used, it and its accessories as well as the packaging material should be sorted correctly.

All important technical information is to be found on the label of the P402, P408 and P421B.

- 1: Manufacturer's name.
- 2: Article number
- 3: Product name
- 4: Year of manufacture
- 5: Month of manufacture
- 6: Max. user weight

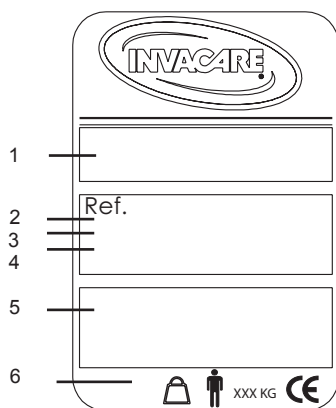

P402: 48 cm
P408: 58 cm
P421B: 62 cm

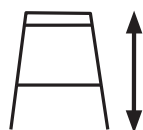

P402: 77-95 cm
P408: 78,5-88,5 cm
P421B: 79-92 cm




Manufacturer:
INVACARE REA AB, Växjövägen 303, SE-343 75 Diö, Sweden
TEL +46 (0)476 535 00, FAX +46 (0)476 535 99
Sales companies:
Invacare Ltd, Pencoed Technology Park, Pencoed, UK-Bridgend CF35 5HZ
TEL (44) (0)1656 776 200, FAX (44) (0)1656 776 201
UK@invacare.com, eire@invacare.com

1534946-2 100419




P402 AVENTIA, P408 MAESTRIA, P421B FORIA


P402 Aventia

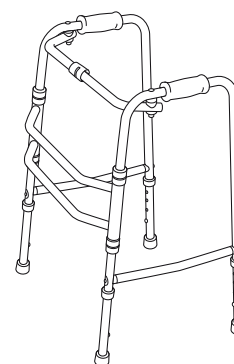

P408 Maestria

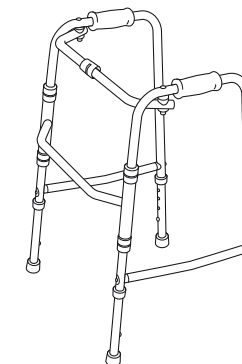

P421B Foria

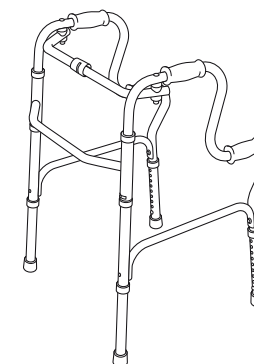

OPERATING INSTRUCTIONS

Read carefully through the operating instructions before use.
The walking frame is adapted for both indoor and outdoor use.

INTENDED USE

The walking frame is intended to be used solely as a walking aid, all other usage can entail risk of injury.

NB!

The content of these operating instructions shall be communicated to the user before the walking frame is handed over.



The prescriber must ensure that the user has sufficient intellectual and physical abilities to use the walking frame.

EN OPERATING INSTRUCTIONS



SAFETY



- Check that the walking frame has been delivered in perfect condition before you use it
- The walking frame must be fully unfolded when used as a walking support. NB. Only to be used as a walking support.
- NB! Do not sit on the crossbar.
- NB! Do not use the crossbar as support.
- Exercise caution when walking on sloping surfaces.
- Do not go up or down stairs with the walking support.
- Do not use the walking support on an escalator.
- Do not exceed the maximum user weight that is specified on the walking support's label.
- Parts that are worn out, damaged or missing must be changed immediately by a qualified technician.
- Do not leave or store the walking support outside.
- Check the stability of the walking support at regular intervals and when necessary that nuts and bolts are tight and that the ferrules are in good condition.
- Exercise great care when using the walking support on wet or slippery surfaces.




Do not sit on the crossbar because of the tipping risk!!

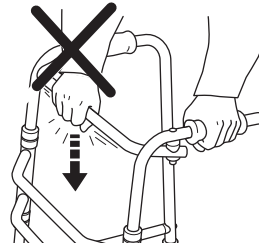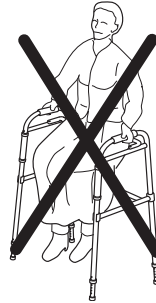

ADJUSTING THE HEIGHT OF THE WALKING SUPPORT

Press in the sprung locking pin until the leg is released. Set the leg at the required length and check that the locking pin is projecting out into the tube. All legs should have the same length.

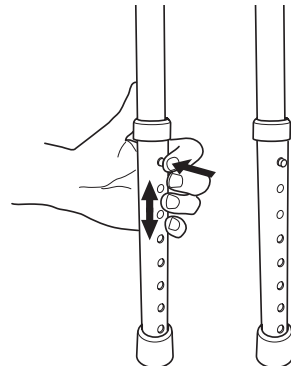


BEFORE USE

All packaging should be removed before using the walking support for the first time.


UNFOLDING

Unfold the side frames until a distinct clicking sound is heard. Check that it is not possible to fold the side frames in before using the walking support.

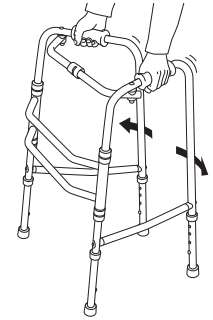

FOLDING


Press the locking knob in the middle of the upper cross bar and fold in the side frames.

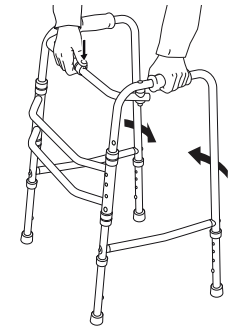

NB!



Do not engage the locking knob unless you intend to fold the walking frame.

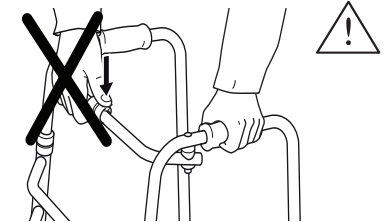

NB!



NB! Be aware of the risk of crushing between the parts of the frame.

