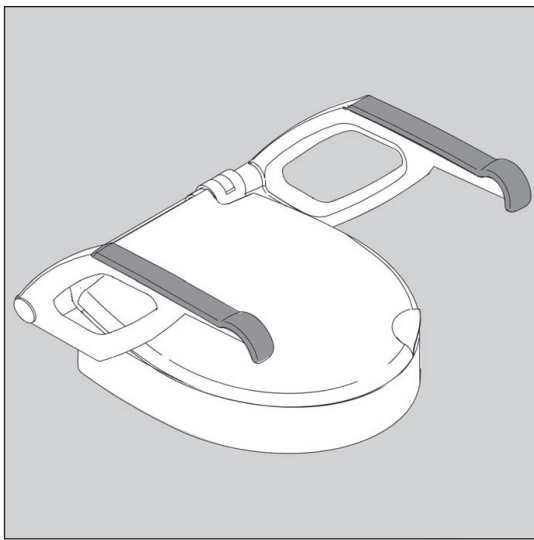
Aquatec® 90000

- EN** Raised toilet seat
User Manual
- DE** Toilettensitzerhöhung
Gebrauchsanweisung
- FR** Rehausse WC
Mode d'emploi
- IT** Rialzo per WC
Istruzioni per l'uso
- ES** Elevador de inodoro
Manual de instrucciones
- NL** Toiletblilverhoging
Gebruiksaanwijzing
- DA** Toiletforhøjelse
Brugsanvisning
- FI** WC-istuimen korotus
Käyttöohje
- NO** Toalettseteforhøyelse
Brukerveiledning
- SV** Toalettstolsförhöjare
Bruksanvisning
- PT** Assento de sanita levantado
Manual de instruções

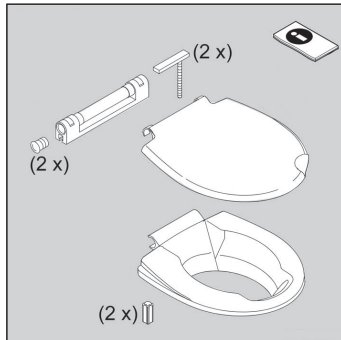
Fig. 1.1

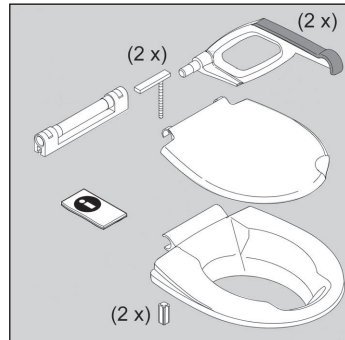
Fig. 1.2

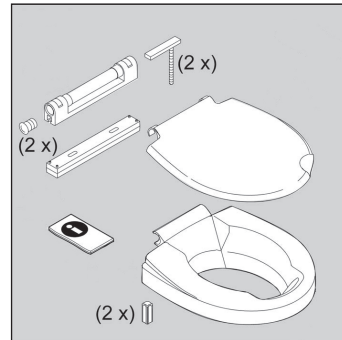
Fig. 2.1

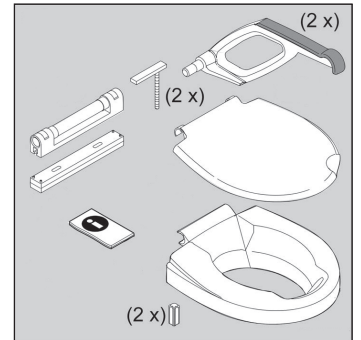
Fig. 2.2

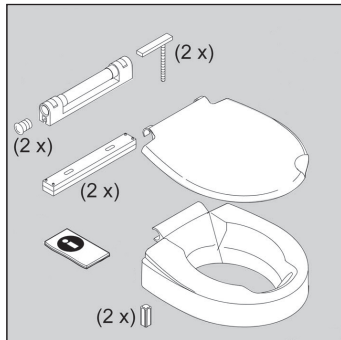
Fig. 3.1

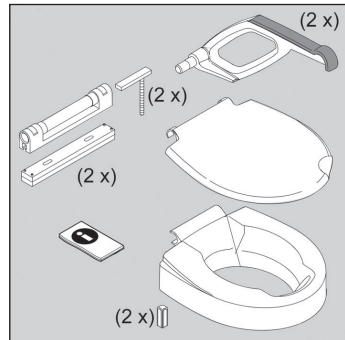
Fig. 3.2

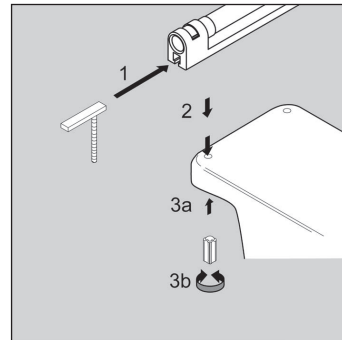
Fig. 4.1

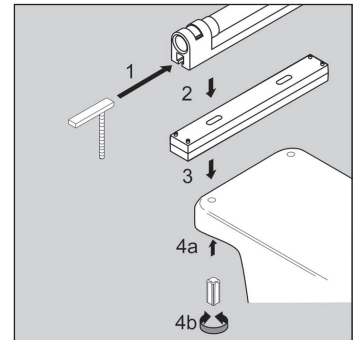
Fig. 4.2

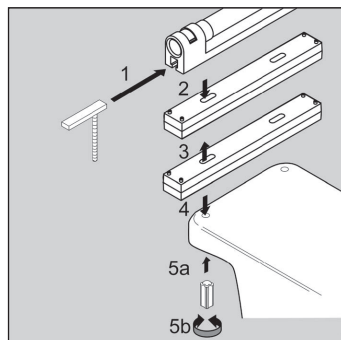
Fig. 4.3

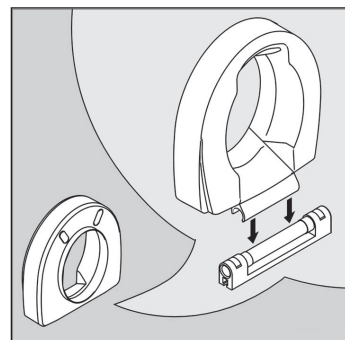
Fig. 5

Fig. 6

This manual must be given to the user of the product.
Before using the product, read this manual and save it
for future reference!

AQUATEC®

Yes, you can.®

Fig. 7.1a

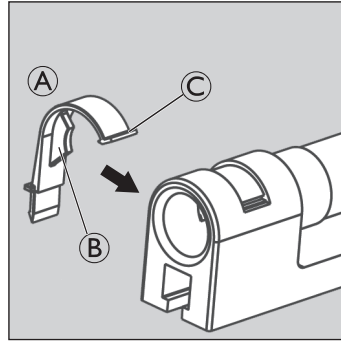
Fig. 7.1b

Fig. 7.2

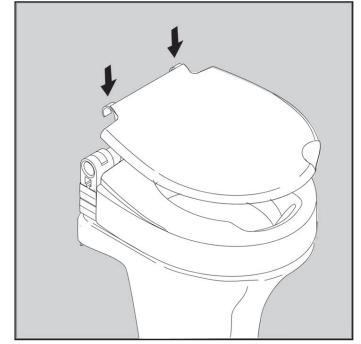
Fig. 8

Intended Use

The product serves exclusively to raise the toilet seat and is therefore only intended for installation on toilets. The purpose of the armrests is to support the natural movement of the user when standing up and sitting down. Any other use is prohibited.

WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

WARNING Risk of falling!

The armrests are intended to be used on both sides evenly. If pressure is applied on one side only, use only half of the body weight. At overload, there is a risk that the armrest will break.

Temperature for use: 10 - 40 °C

Max. user weight: 150 kg

Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

WARNING Risk of injury!

- Do not use if defective.

Scope of delivery

Height	without armrests	with armrests
2 cm	▸ Fig. 1.1	▸ Fig. 1.2
6 cm	▸ Fig. 2.1	▸ Fig. 2.2
10 cm	▸ Fig. 3.1	▸ Fig. 3.2

Assembly

CAUTION Risk of trapping!

- Be careful during assembly.

Height 2 cm	▸ Fig. 4.1
Height 6 cm	▸ Fig. 4.2
Height 10 cm	▸ Fig. 4.3

▸ Fig. 5 and 6.

with armrests	▸ Fig. 7.1a ▸ Fig. 7.1b
---------------	----------------------------

- Ensure that the red clips (A) engage at two points: first the pin (B) and then the upper end of the clip (C).
- To disassemble the armrest, first release the upper end of the clip (C) and then the pin (B).

without armrests	▸ Fig. 7.2
------------------	------------

▸ Fig. 8.

Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 85 °C for three minutes.

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

- Check product regularly for damage and to ensure that it is securely assembled.

Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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Algol-Trehab Oy
Aquatec Operations GmbH