DOLOMITE

DOLOMITE *HP*

SV BRUKSANVISNING

NO BRUKSANVISNING

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GB OPERATING INSTRUCTIONS

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FR MANUEL D' UTILISATION

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DOLOMITE HP

SV BRUKSANVISNING

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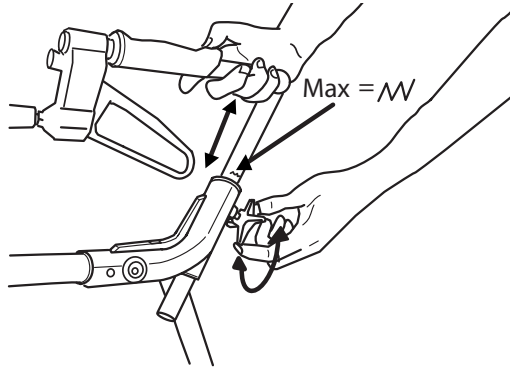
DOLOMITE HP

GB HANDLEBARS

For adjustment; Loosen the wring, adjust the handle to suitable height and tighten the wring properly. Therapist may suggest different heights depending on specific need of the client.

By heavier load on the HP-handle, we recommend replacement of the height adjustment knob with an Allen screw (art no 60140) to gain better torque.

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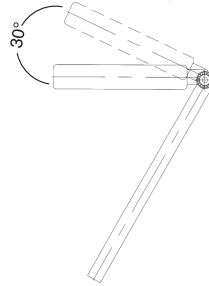
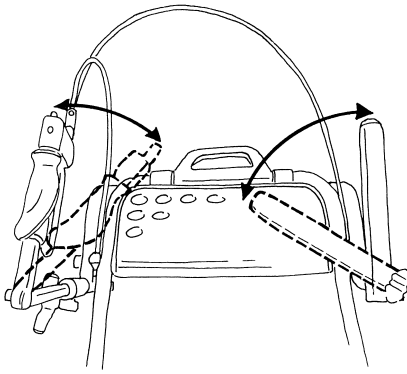
GB HANDLEBARS

The handles can be angled maximum 45° towards the center of the walker. Arrow pointing on the thin inner vertical line indicates maximum allowed outer handle position. Handles should be placed horizontally maximum 30° upwards

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Håndtak kan vinkles maksimum 45 grader innover. Håndtaket er innstilt i maksimum. Når pilen peker på den indre tynne vertikale linje, indikeres maks tillat ytre håndtaksstilling. Håndtakene kan vinkles maksimum 30° oppover fra horisontal stilling.

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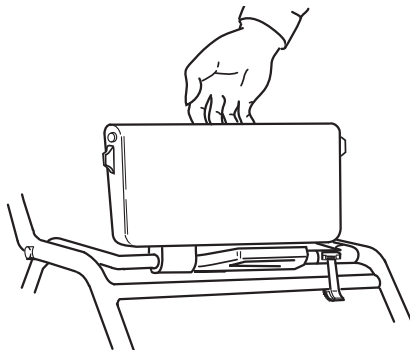
GB FOLDABLE SEAT AND FOLDING

The seat is foldable for increased space between the handles.

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All technical data available on the **DOLOMITE HP** plus label:

- | | | | |
|-----------|--------------------|--------------------------|-----------------------------|
| GB | 1: Article number | 4: Date of manufacture | 7: In- and outdoor rollator |
| | 2: Name of product | 5: Maximum user's weight | 8: Max. width of rollator |
| | 3: Serial number | 6: NB: See user manual | 9: Manufacturer's name |

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GB INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:

When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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GB MAINTENANCE – SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments are properly tightened.**
- **Immediately contact the nearest dealer if a fault should appear. Never use a defective walker.** Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid! User's max. weight: 275 lbs. (125 kg, 20 stones.) Dolomite will **not** accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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Invacare A/S, Sdr. Ringvej
Invacare Poirier SAS
Algol-Trehab Oy Karapellontie 6