
Invacare® Stand up support

Accessories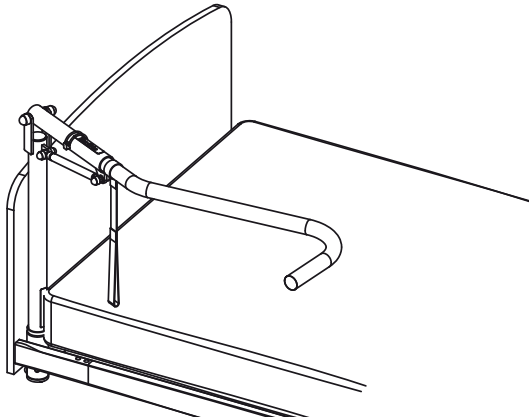

User manual
Brugsanvisning
Bruksanvisning
Brugsvejledning
Gebrauchsanweisung
Gebruiksaanwijzing
Manuel d'utilisation
Manual del usuario
Manuale d'uso

EN

DA

SV

NO

DE

NL

FR

ES

IT

This manual must be given to the user of the product.
Before using this product, read this manual and save for future reference

Yes, you can.®

Stand up support

Application:

The stand up support can be mounted in both sides at the bed, so the user easily can get to a standing position and get in or out of the bed.

If the stand up support is not in use, it can be raised to a vertical position.

The stand up support must not be loaded with more than 60 kg downwards and must not be loaded in the horizontal direction away from the bed, thereby avoid the risk of tipping the bed.

Always turn the castors pointing outwards to make the bed more stable and brake at least 2 castors of the bed


Make sure that the stand up support don't touch the ceiling, when lifting the bed to top position.

Use:

- Do not load the handle in any other direction, than shown in fig. A - (max. 60 kg downwards)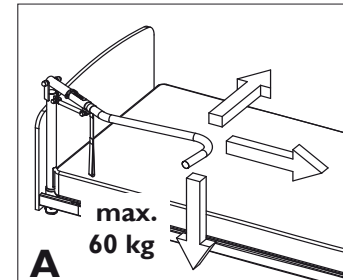





Installation:

The stand up support is mounted as the lifting pole.

1. Remove the plastic cap from the bed lifting tube.
2. Mount the stand up support in the lifting pole tube (fig. B) - oriented as shown on fig. D
3. Elevate the bed to top position for better access to the underside.
4. The disc must be mounted at the bottom of the stand up support pole with 2 screws. (C)
5. Tighten the screws with the enclosed Allen-Key.

NOTE! Make sure that both screws are tightened.

- Correct mounted (fig. D) - The stand up support must be mounted along the bed with the handle pointed away from the bed.
- Incorrect mounted (fig. E and F) - The stand up support must not be mounted so the handle pointed in or turned away from the side of the bed.

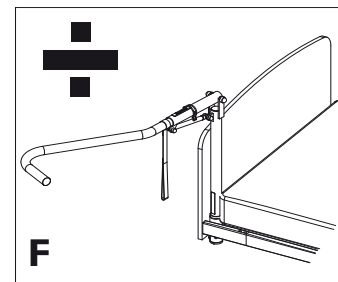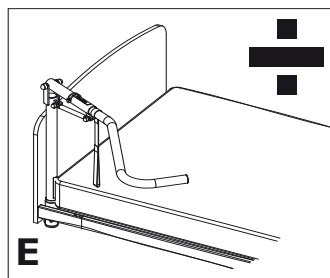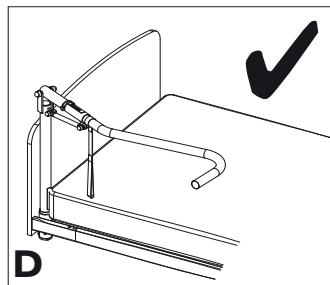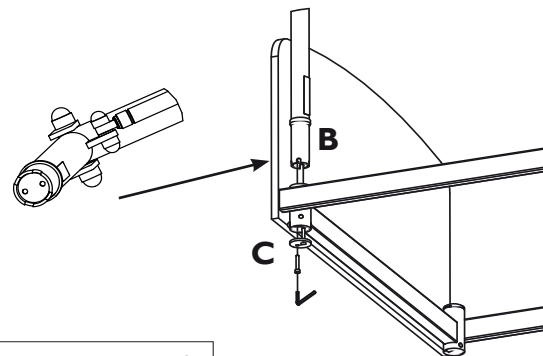Application:

The stand up support can be used on the following bed types:

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Switzerland INVACARE AG