

### Mounting the Rigging screw - Azalea




**WARNING!**

Safety risk


The wheelchair may collapse

– Remember to always reinsert and fasten the safety pin when it has been removed.


– Check that the lock shackle / loop is securely locked.

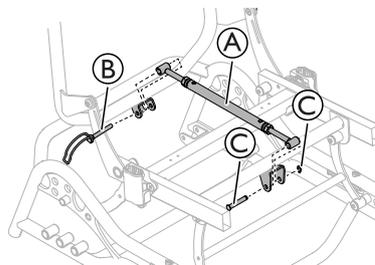


1. Attach the Rigging screw (A) in the front and the rear attachments.
2. Fixate the Rigging screw (A) in the rear attachment with the safety shackle (B).

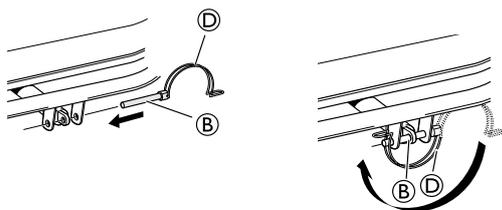

3. Close the safety shackle (B) with the lock shackle (D) to secure the Rigging screw (A).
4. Fixate the Rigging screw (A) in the front attachment with the safety pin and the locking washer (C).

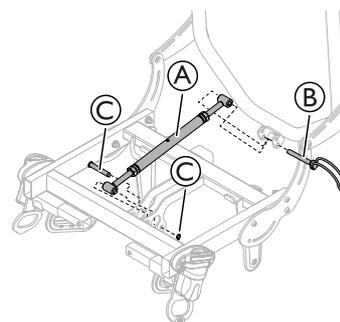

1. Attach the Rigging screw (A) in the front and the rear attachments.
2. Fixate the Rigging screw (A) in the rear attachment with the safety shackle (B).

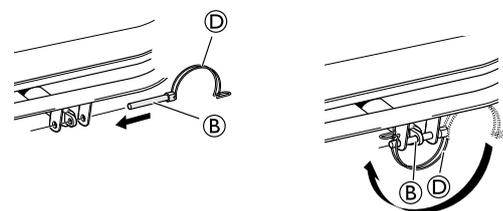

3. Close the safety shackle (B) with the lock shackle (D) to secure the Rigging screw (A).
4. Fixate the Rigging screw (A) in the front attachment with the safety pin and the locking washer (C).

**Note**

This instruction is an extract from the service manual. It is available in multiple languages and can be downloaded from the Invacare website. Please contact the Invacare representative in your country.

### Mounting the Rigging screw - Dahlia



**WARNING!**

Safety risk

The wheelchair may collapse

– Remember to always reinsert and fasten the safety pin when it has been removed.

– Check that the lock shackle / loop is securely locked.



Invacare Rea AB  
Växjövägen 303  
SE-343 71 Diö  
Sweden



Yes, you can.®