



**Serial number: P15** \_\_\_\_\_

**Specialist dealer:** \_\_\_\_\_

**Alber GmbH**

Vor dem Weißen Stein 21

72461 Albstadt, Germany

Phone +49 (0)7432 2006-0

Fax +49 (0)7432 2006-299

WORK	CHECK	REMARKS
<b>Drive wheel</b>		
Cleaning the drive wheel	<input type="checkbox"/>	
Check tyre for damage	<input type="checkbox"/>	
Check the profile depth of the tyre	<input type="checkbox"/>	min. 0,5 mm
Check air pressure	<input type="checkbox"/>	2 bar
Check spokes for tight fit and completeness	<input type="checkbox"/>	
Check brake	<input type="checkbox"/>	
Check brake pads	<input type="checkbox"/>	< 0,5mm replace
Check brake disc	<input type="checkbox"/>	Discoloration/wear/deformation
Check drive wheel for noise	<input type="checkbox"/>	
Check actuator plug connection for tight fit	<input type="checkbox"/>	
Check wheel axle nuts	<input type="checkbox"/>	35 Nm
<b>Display</b>		
Remove and clean display	<input type="checkbox"/>	
Clean contact surfaces at the bottom of the display	<input type="checkbox"/>	with a damp cloth
Check housing incl. display screen for damage	<input type="checkbox"/>	
Check correct display	<input type="checkbox"/>	
<b>Display holder</b>		
Check screw on holder	<input type="checkbox"/>	
Clean holder and contacts	<input type="checkbox"/>	
Check spring pins for function	<input type="checkbox"/>	
Check cable for damage	<input type="checkbox"/>	
<b>Control elements</b>		
Check brake lines for damage	<input type="checkbox"/>	
Check brake lever for damage and function	<input type="checkbox"/>	
Check lighting and horn for function	<input type="checkbox"/>	
Check cable operating elements for damage	<input type="checkbox"/>	
<b>Battery attachment</b>		
Cleaning the battery attachment	<input type="checkbox"/>	
Check contact between battery and attachment	<input type="checkbox"/>	
Detent battery/lock check for function	<input type="checkbox"/>	

**Serial number: P15** \_\_\_\_\_

**Specialist dealer:** \_\_\_\_\_

**Alber GmbH**

Vor dem Weißen Stein 21  
 72461 Albstadt, Germany  
 Phone +49 (0)7432 2006-0  
 Fax +49 (0)7432 2006-299

WORK	CHECK	REMARKS
<b>Battery pack</b>		
Cleaning the battery pack incl. charging socket	<input type="checkbox"/>	
Check housing for damage	<input type="checkbox"/>	
Check charging socket cover	<input type="checkbox"/>	
Charge the battery	<input type="checkbox"/>	70% of the total capacity
Check charge status via key	<input type="checkbox"/>	
<b>Frame / Fork</b>		
Check frame and welds for damage	<input type="checkbox"/>	
Check steering for play and function	<input type="checkbox"/>	
Check reset of steering, clean magnet	<input type="checkbox"/>	
Check stand, wheels and rubber connectors	<input type="checkbox"/>	
Check torque arm for tight fit	<input type="checkbox"/>	
<b>Final inspection</b>		
Check that the bracket/docking strut is firmly seated	<input type="checkbox"/>	
Check docking mechanism for function	<input type="checkbox"/>	
Check charger function	<input type="checkbox"/>	
Function test of complete system with occupant	<input type="checkbox"/>	
<b>Documentation</b>		
Replace the "Maintenance" label on the device	<input type="checkbox"/>	Marking with waterproof fibre pen. In the „Next check“ field, mark the date (interval 24 months)

**!** Maintenance on the e-pilot may only be carried out by trained specialist personnel

\_\_\_\_\_  
Date

\_\_\_\_\_  
Name

\_\_\_\_\_  
Signature of technician