## Invacare® Evacuation Mattress Cover

en	Accessory for mattresses	
	User Manual. . . . .	2
de	Zubehör für Matratzen	
	Gebrauchsanweisung. . . . .	5
fi	Patjojen lisävaruste	
	Käyttöohje. . . . .	9
no	Tilbehør til madrasser	
	Bruksanvisning. . . . .	12
sv	Tillbehör till madrasserna	
	Bruksanvisning. . . . .	15

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and  
saved for future reference.

# 1 General

## 1.1 Introduction

This document is a supplement to the product's user documentation.

This component itself does not bear a CE and a UKCA mark but is part of a product that complies with the Medical Device Regulation 2017/745, Class I and Part II UK MDR 2002 (as amended) Class I concerning medical devices. It is therefore covered by the product's CE and UKCA marking. See the product's user documentation for more information.

Only use this component if you have read and understood this manual. Seek additional advise from a healthcare professional who is familiar with your medical condition and clarify any questions regarding the correct use and necessary adjustment with the healthcare professional.

Note that there may be sections in this document, which are not relevant to your component, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the component.

Invacare reserves the right to alter component specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the component, for example safety notices and recalls, contact your Invacare representative. See addresses at the end of this document.

In case of a serious incident with the component, you should inform the manufacturer and the competent authority in your country.

## 1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

### **WARNING!**

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

### **CAUTION!**

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

### **NOTICE!**

Indicates a hazardous situation that could result in damage to property if it is not avoided.

### **Tips and Recommendations**

Gives useful tips, recommendations, and information for efficient, trouble-free use.

## 1.3 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## 1.4 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

# 2 Safety

## 2.1 Safety Information

### **CAUTION!**

#### **Risk of Injury or Damage to Property**

- Mattress must be used with the cover at all times.

# 3 Product Overview

## 3.1 Product Description

The Invacare Evacuation Mattress Cover is a product designed to allow a quick response in the event of an evacuation of the building. This mattress cover has a built-in quick release webbing which is used to secure the patient in the mattress safely. The head and foot end straps have quick release extendable pull straps to drag both the mattress and the patient safely, eliminating any trip hazard when pulling the patient out of the building.

The Invacare **Evacuation Mattress Cover** is a single use product. After an emergency evacuation it **MUST** be discarded and substituted by a new one.

## 3.2 Scope of Delivery

The following components are included within the scope of delivery:

(A)	Evacuation Mattress Cover
(B)	User Manual

## 4 Setup

### 4.1 Preparing the Evacuation Mattress Cover for use

The following steps must be followed to prepare the mattress for evacuation.


#### WARNING!

##### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage. Before initiating an evacuation procedure:

- Ensure the patient is positioned in the middle of the mattress and that there are no part of the body overhanging the mattress.

1. Pull strap away from the securing tabs until the strap is fully extended.

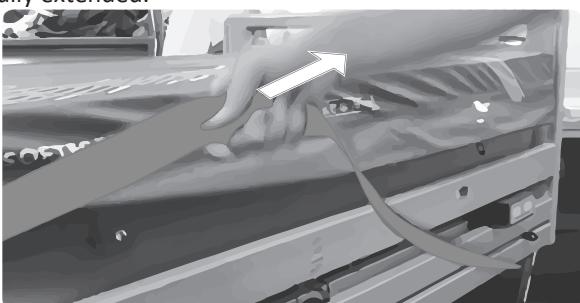2. Throw the extended strap across the mattress and patient.

3. On opposite side of the mattress, release the opposite strap.

4. Pass the securing strap across the torso of the patient, loop the strap through the handle at the side of the mattress.
5. Pull strap firmly across the patient ensuring the strap is secure across the chest and the patient arms are under the straps and not over. Mate the hook and loop together on securing strap.

6. Repeat steps 1 through to 5 for opposite side on mattress ensuring the straps are secured across the patient thighs.
7. When the patient is securely held in the mattress you can release the drag straps located at the head and foot end of the mattress.

8. Pull the drag straps away for the retaining tabs to extend the drag strap.

## 5 Usage

### 5.1 Emergency Evacuation


#### WARNING!

##### Risk of Serious Injury or Damage


- Improper use of this product may cause injury or damage. Before initiating an evacuation procedure:
- **ALWAYS** lower the bed to its lowest position.
  - Ensure the patient is positioned in the middle of the mattress and that there are no part of the body overhanging the mattress.

#### Horizontal Evacuation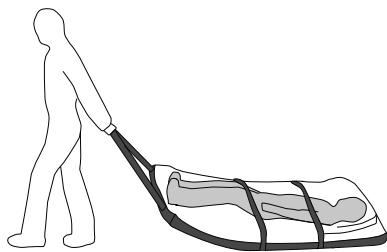

Take a firm hold of the loop drag handle at the foot end, allowing you to evacuate the patient in a dragging motion. It is advisable that a second person uses the drag strap at the head end to navigate more easily around bends.

#### Vertical Evacuation

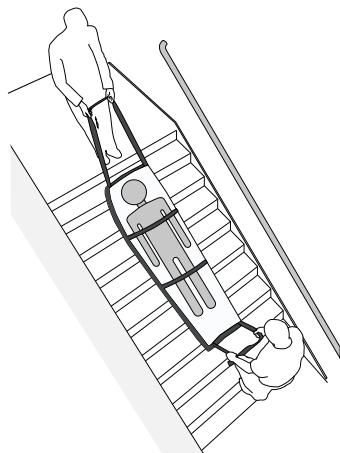If the patient is located on a higher or lower level, you will need to use the stairwells to make your way to the ground floor. Make sure you have enlisted a second person to assist and ensuring you take command of the stairs, the second person can be responsible for any necessary breaking when making your way down the stairs.


### 5.2 Reset the Product

- The following reset procedure should only be performed if any of the straps or retainers have been inadvertently detached.

#### Securing straps

1. Release the securing straps by unfastening the hook and loop strap across the patient area, and detach the straps from the handles on the sides of the mattress. Ensure the straps are not twisted.
2. Push each securing strap back into its original position:
  - Fold the strap neatly, locking it in place with the snap fastener **A**.
  - Press it firmly so that the hook and loop strap connects securely **B**.
  - Repeat this process on both sides of the mattress.

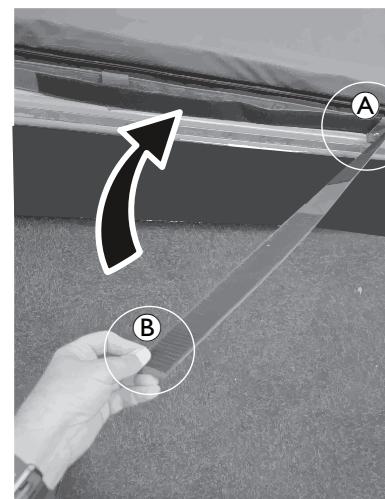3. Wrap the loop snap fastener **C** around the hook and loop strap, and close it.


## Drag straps


- Push the drag straps back into their original position by folding and aligning them properly on both sides of the mattress. Connect the black snap fasteners **D** to eliminate any slack.

- Close the loop snap fastener **E** to secure the strap. Repeat this process on both sides of the mattress.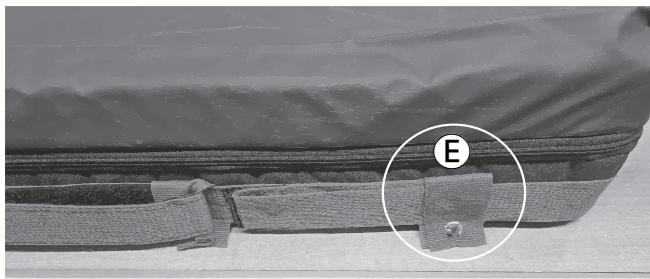

- Complete the process by closing both snap fasteners at the head and foot ends.

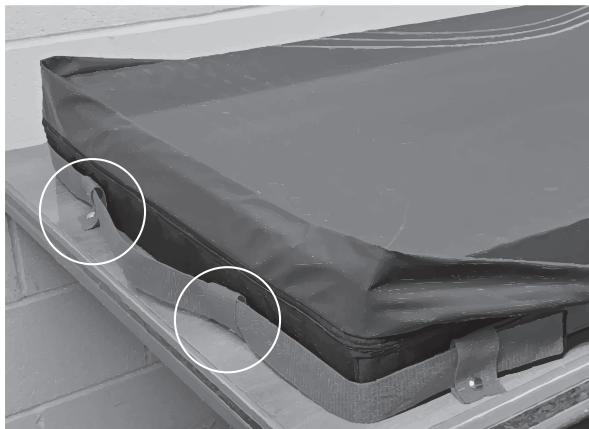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3. Vira ögletryckknappen © runt kardborreremmen och stäng den.
Invacare AB
Invacare Limited
Invacare UK Operations Limited