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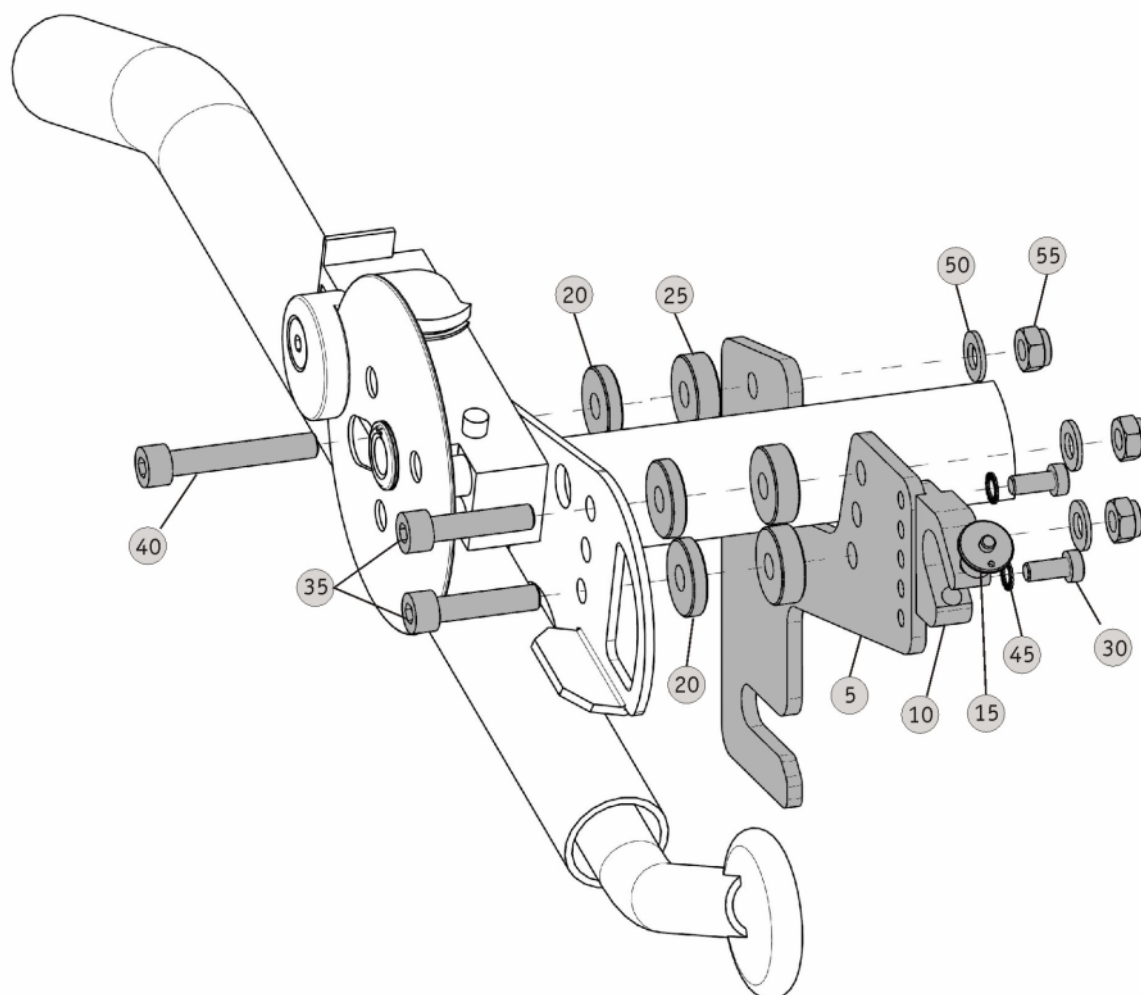
[Mounting suggestion; consignment may contain material for different mounting variants.]

[Please note that the bracket may slightly differ from the drawing.]

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[**Caution**, any mounting must be done by personnel authorized by the manufacturer !]

Montageanweisung [Mounting instruction]

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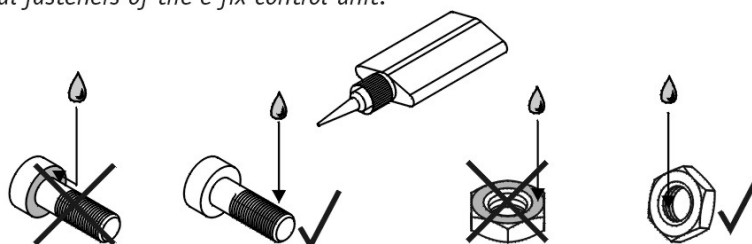
Locking of screws and screw nuts

If there is no other specification in our mounting instruction: Save all screw fittings which have no locking washer with screw locking adhesive (see pictures below). Use WEICON adhesive AN 302-43 (blue) / LOCTITE 243 adhesive to stick adapter sockets and screws. Use WEICON adhesive AN 306-01 (green) / LOCTITE 601 adhesive to stick the mechanical fasteners of the e-fix control-unit.

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Allowed wheel camber of the e-fix and e-motion

Don't pass over the following maximum values, if you mount the bracket at a wheelchair which offers the adjustemt of the wheel camber.

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The numerical datas in the charts above replace the specifications of the previous pages.

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Fastening torques for screw fittings

Unless otherwise noted, seize the screw fittings as shown in the following table:

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[Caution, any mounting must be done by personnel authorized by the manufacturer !]

Montageanweisung [Mounting instruction]

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Important Liability Note

A combination of the mounting systems introduced by Alber GmbH (Alber) together with auxiliary drives not manufactured by Alber is not authorized by Alber. Such combination does not comply with the intended use to be defined in accordance with the medical device directive (MDD) and therefore constitutes non-conformity according to the MDD.

By way of precaution, Alber GmbH rejects any liability regarding warranty services or possibly resulting property damages and personal injuries due to a combination of Alber mounting systems with third party brands.

Pos. [Pos.]	Menge [Quantity]	Einheit [Unit]	Art.-Nr. [Art.-No.]	Lagerplatz/Alber [Stock/Alber]	Bezeichnung [Description]
005	2	St [Pcs]	1564899	PROD	Halterung HD Balance Handwerksdesign [Bracket HD Balance Handwerksdesign]
010	2	St [Pcs]	1463160	719/03/007-009	Standardgabel blau vz [Standard fork blue galvanized]
015	2	St [Pcs]	1456406	721/02/007-008	Quickpin Sicherungsstift [Quickpin quick release bolt]
020	6	St [Pcs]	1456632	1469/04/002	Distanzring Alu 25 x 8,4 x 5 lang [Spacer ring aluminium 25 x 8,4 x 5]
025	6	St [Pcs]	1456633	1469/04/003	Distanzring Alu 25 x 8,4 x 8 lang [Spacer ring aluminium 25 x 8,4 x 8]
030	4	St [Pcs]	1463716	1465/02/002	Zylinderschraube DIN 7984 - M 6 x 12 - 8.8 vz [Cylinder head screw DIN 7984 - M 6 x 12 - 8.8]
035	4	St [Pcs]	1459412	1463/04/006	Zylinderschraube DIN 912 - M 8 x 35 [Cylinder head screw DIN 912 - M 8 x 35]
040	2	St [Pcs]	1459421	1463/05/003	Zylinderschraube DIN 912 - M 8 x 50 - 8.8 vz [Cylinder head screw DIN 912 - M 8 x 50 - 8.8]
045	4	St [Pcs]	1456223	1468/00/003	Sicherungsscheibe S - 6 - FSt [Securing disk S - 6 - FSt Schnorr]
050	6	St [Pcs]	1456184	790/02/003	Scheibe DIN 125 [Washer DIN 125]
055	6	St [Pcs]	1456205	790/01/004	Sechskantmutter [Hexagon nut DIN 985 - M 8 - 8 vz]

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