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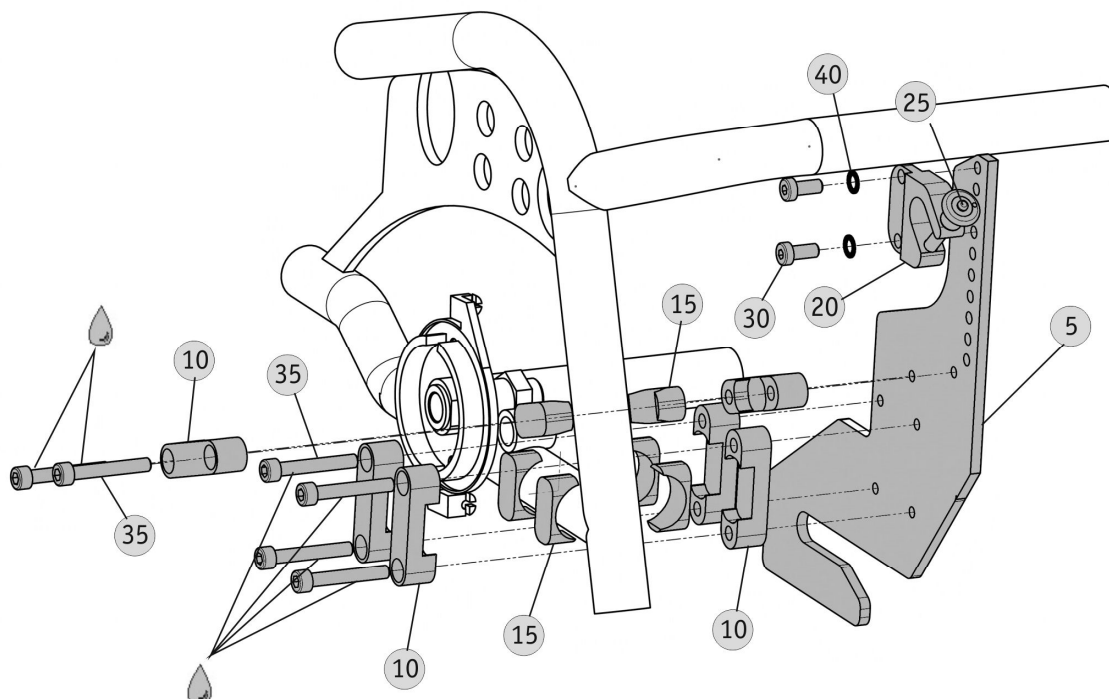
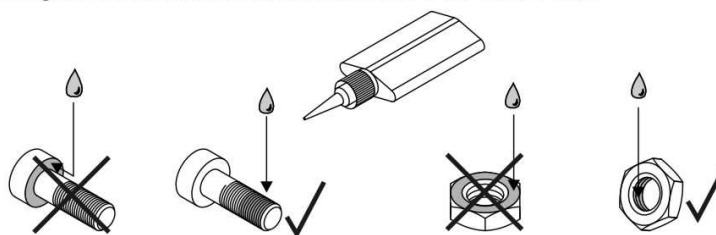
Locking of screws and screw nuts

If there is no other specification in our mounting instruction: Secure all screw fittings which have no locking washer with screw locking adhesive (see pictures below). Use WEICON adhesive AN 302-43 (blue) / LOCTITE 243 adhesive to glue adapter sockets and screws. Use WEICON adhesive AN 306-01 (green) / LOCTITE 601 adhesive to glue the mechanical fasteners of the e-fix control-unit.

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Allowed wheel camber of the e-fix and e-motion

Do not exceed the following maximum values, if you mount the bracket at a wheelchair which offers the

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Fastening torques for screw fittings

Unless otherwise noted, tighten the screw fittings as shown in the following table:

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Important Liability Note

A combination of the mounting systems introduced by Alber GmbH (Alber) together with add-on drives not manufactured by Alber is not authorized by Alber. Such combination do not comply with the intended use to be defined in accordance with the medical device directive (MDD) and therefore constitutes non-conformity according to the MDD.

By way of precaution, Alber GmbH rejects any liability regarding warranty services or possibly resulting property damages and personal injuries due to a combination of Alber mounting systems with third party brands.

Pos. [Pos.]	Menge [Quantity]	Einheit [Unit]	Art.-Nr. [Art.-No.]	Lagerplatz/Alber [Stock/Alber]	Bezeichnung [Description]
05	2	St	1566406	717/02/017-018	Halterung IVC Azalea [bracket set IVC Azalea]
10	12	St	1467022	707/04/013-014	Klemmstück [Clamp colourless anodized]
15	12	St	1467018	707/03/003	Einlegeprofil 25,5 mm [Inlet profile 25,5 mm colourless anodized]
20	2	St	1463160	719/03/007-009	Standardgabel blau vz [Standard fork blue galvanized]
25	2	St	1456406	721/02/007-008	Quickpin Sicherungsstift [Quickpin quick release bolt]
30	4	St	1463716	1465/07/002	Zylinderschraube DIN 7984 - M 6 x 12 - 8.8 vz [Cylinder head screw DIN 7984 - M 6 x 12 - 8.8]
35	12	St	1459361	790/02/020	Zylinderschraube DIN 912 - M 6 x 50 - 8.8 vz [Cylinder head screw DIN 912 - M 6 x 50 - 8.8]
40	4	St	1456223	1461/04/003	Sicherungsscheibe S - 6 - FSt [Securing disk S - 6 - FSt Schnorr]

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