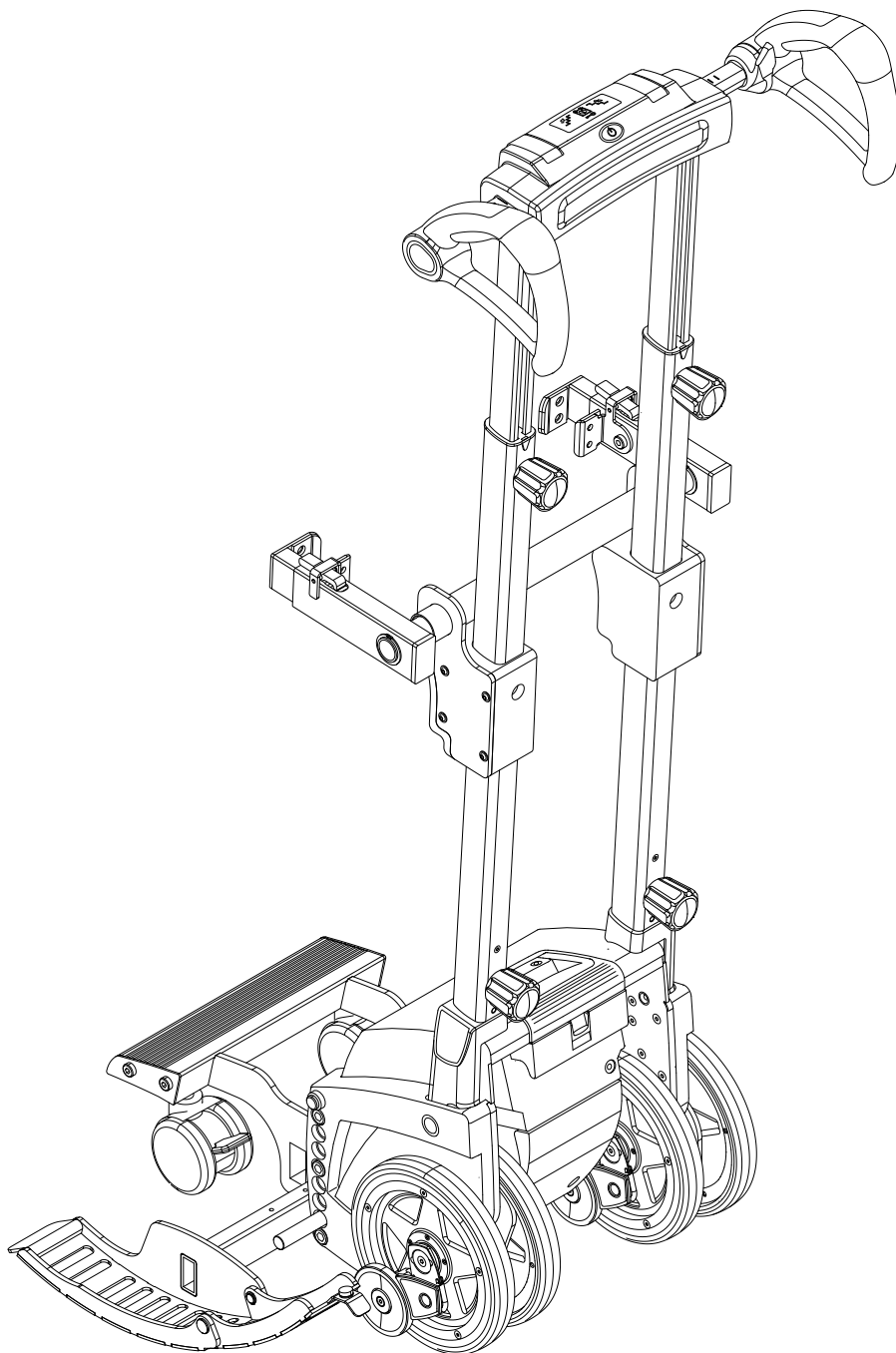
scalaport X7

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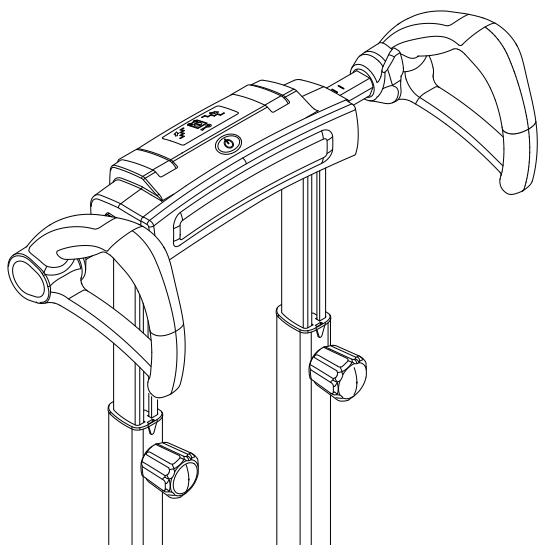
*Instruction for mounting at the scalamobil S35/S38 and the scalamobil S45
and user manual*

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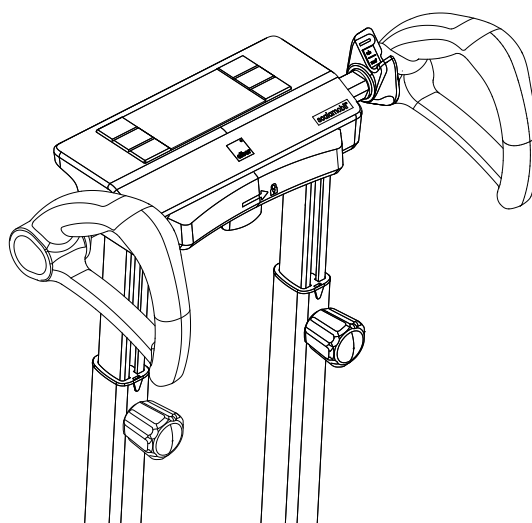
Important notes:

The illustrations in these installation and operating instructions do not necessarily correspond in every detail to the appliance you are using.

We reserve the right to make technical changes.

Mounting the scalaport X7 on the scalamobil S35/S38 is the same as mounting the scalaport X7 on the scalamobil S45.

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Dimensions and weights:

Please refer to the scalaport X7 information brochure for dimensions and weights.

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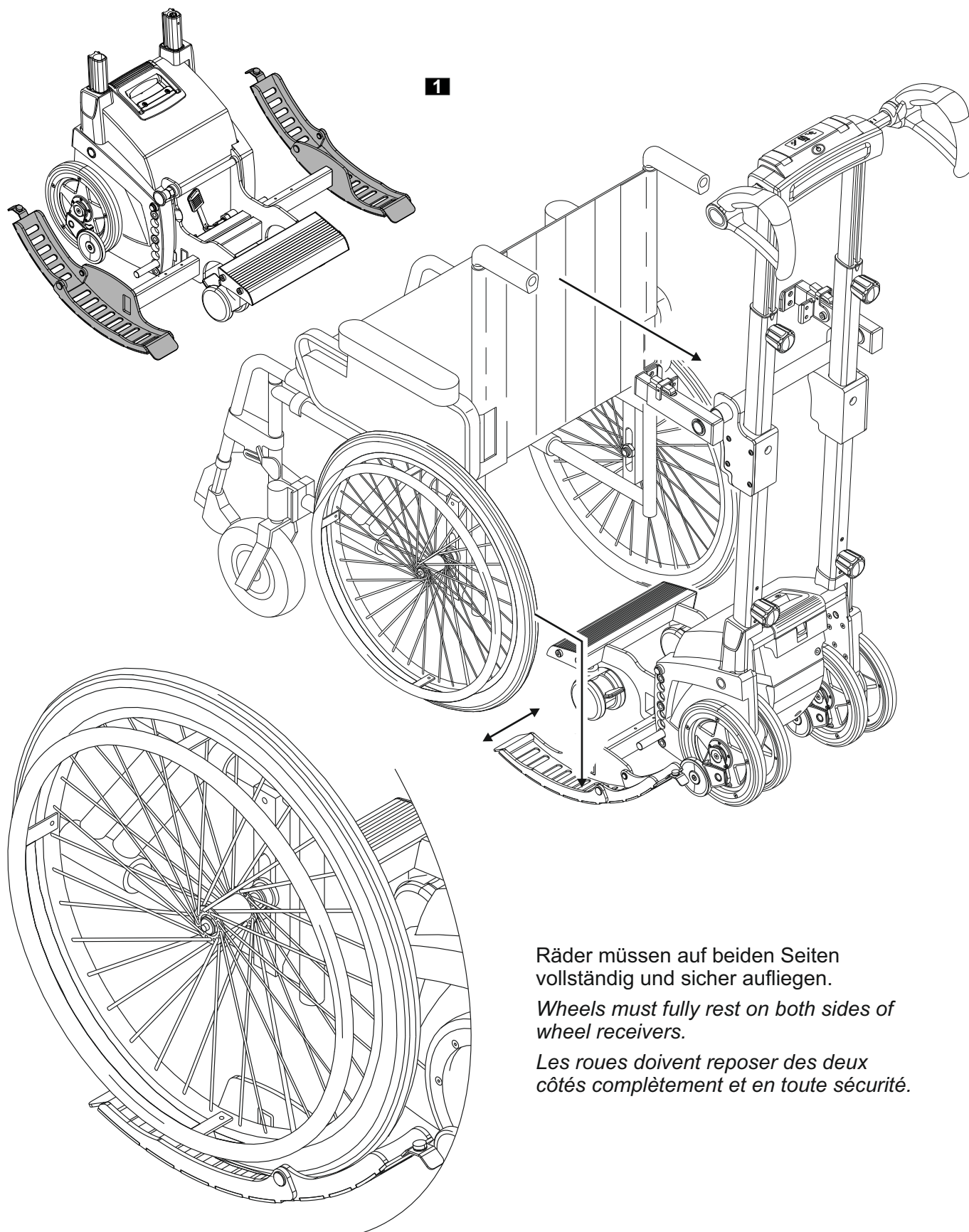
First extent both X7 wheel receivers according to width of wheelbase. Stand behind wheelchair and stairclimber and pull wheelchair backward onto wheel receivers.

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Wheels must fully rest on both sides of wheel receivers.

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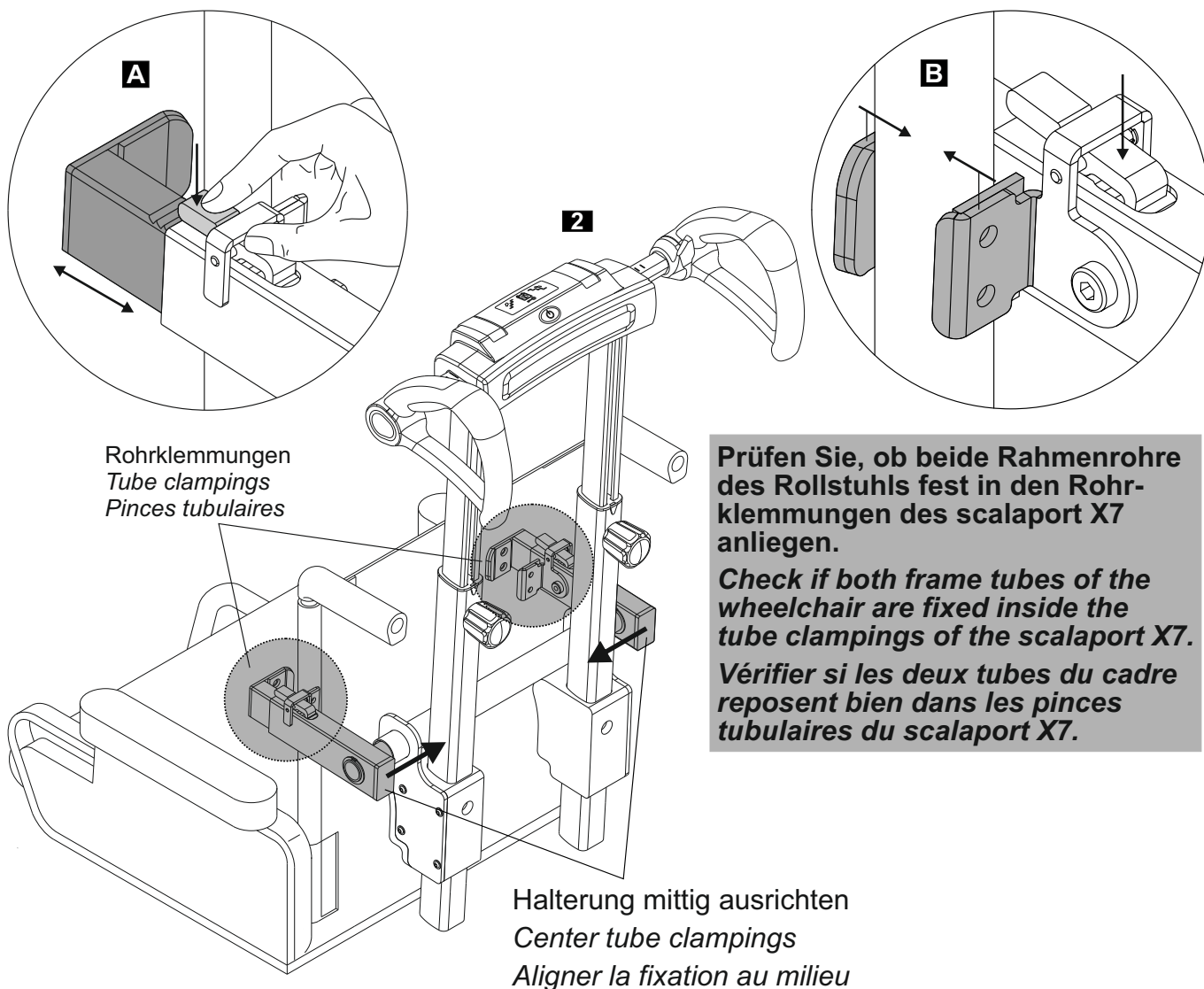
Move the two tube clampings of the scalaport X7 over the left and right tube of the wheelchair's backrest (see drawing A). Then securely fix the two tube clampings (see drawing B).

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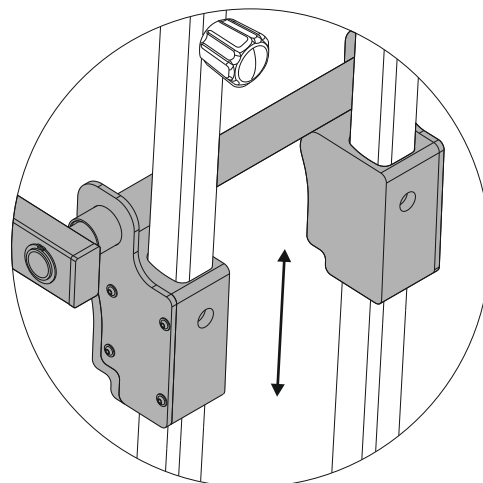
Height adjustment to fit backrest: Slide the clamping mechanism up or down.

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Important note for wheelchairs with an adjustable backrest angle

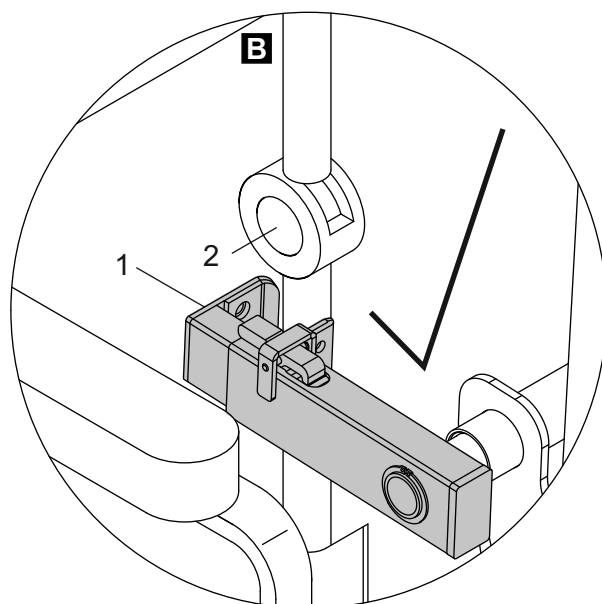
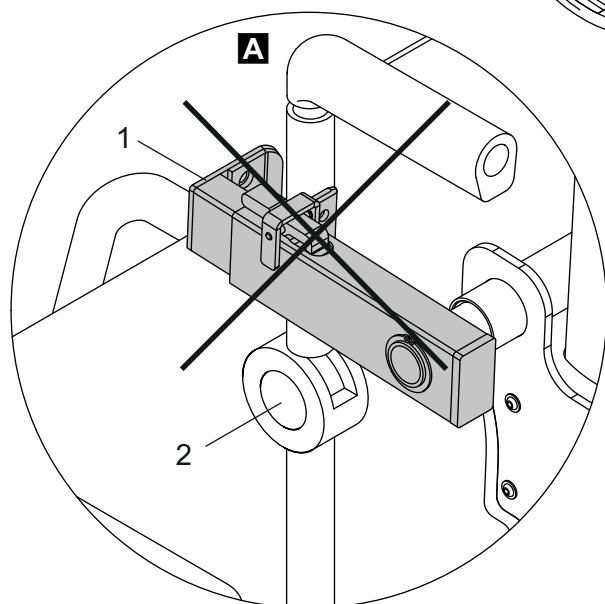
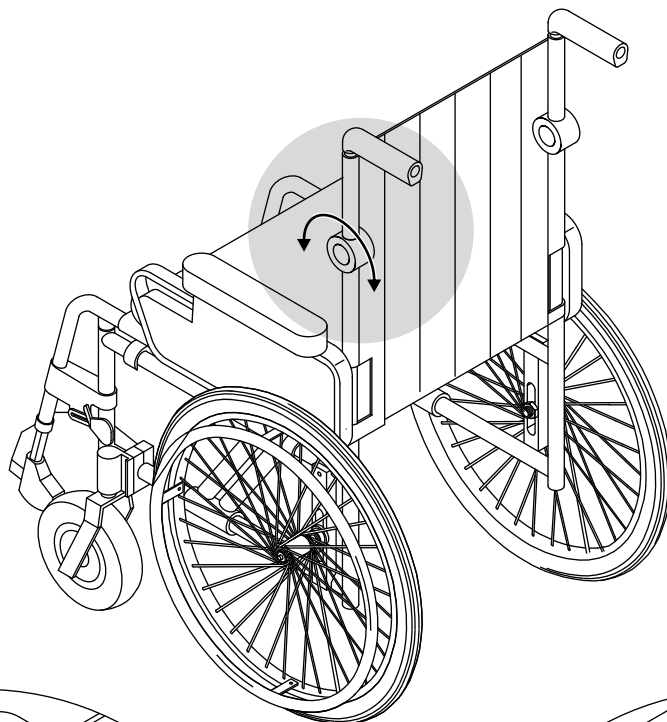
When assembling the scalaport X7, take care when positioning the two tube clamps [1]. These must not be attached above a hinge joint [2], i.e. on the moving part of the backrest (see diagram A).

They may only be assembled underneath one of the hinge joints [2] on the non-moving part of the backrest (see diagram B).

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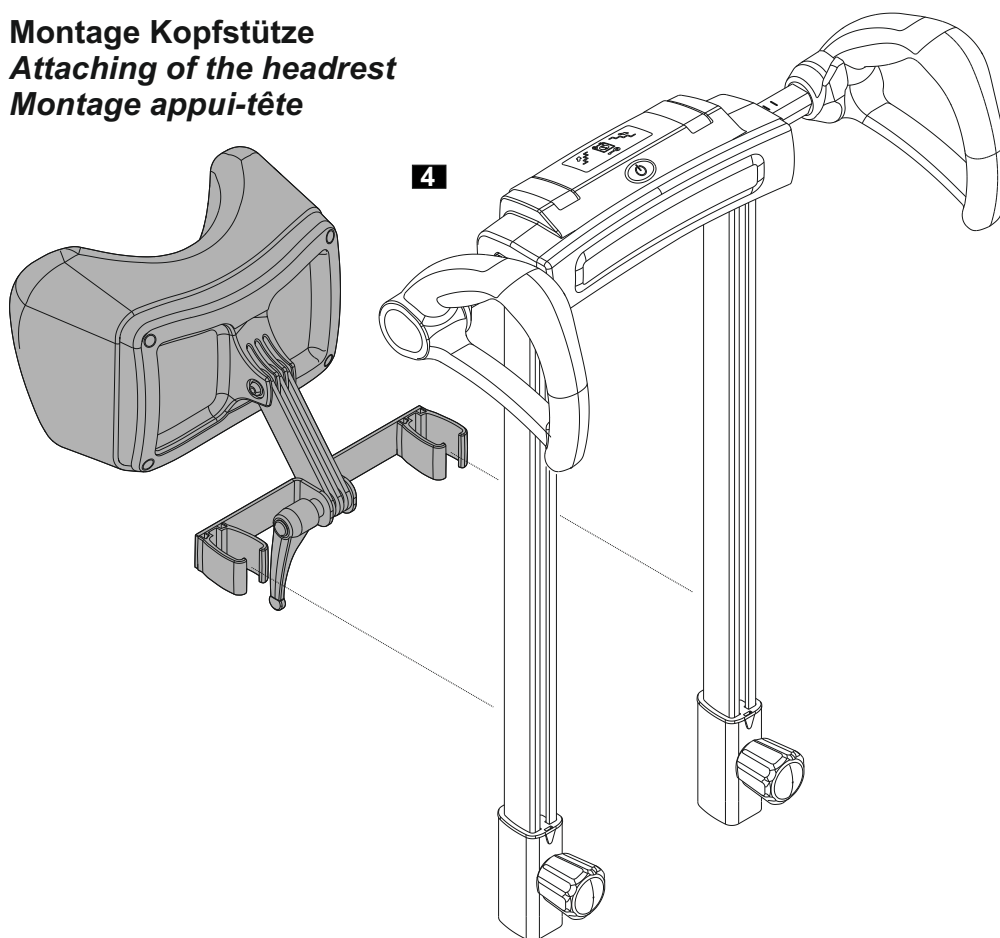
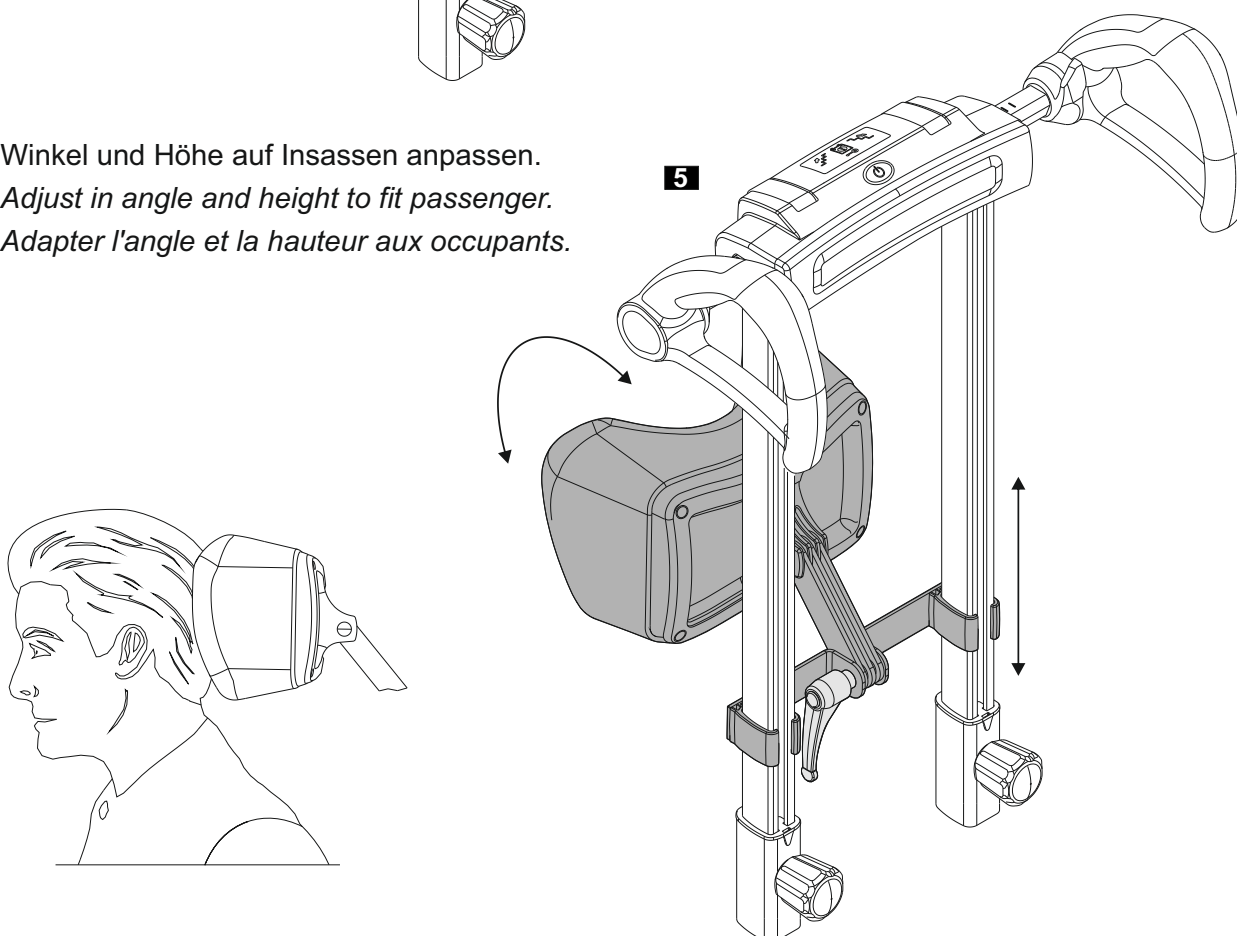
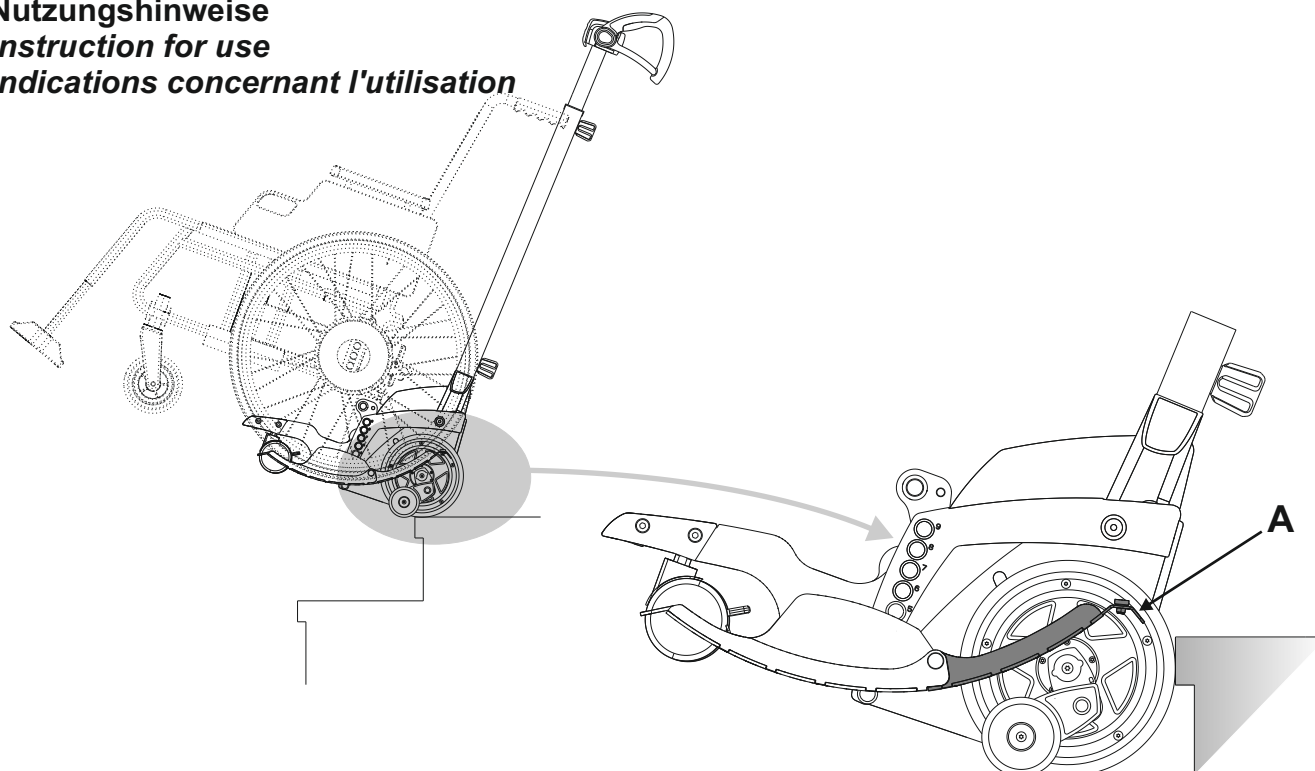
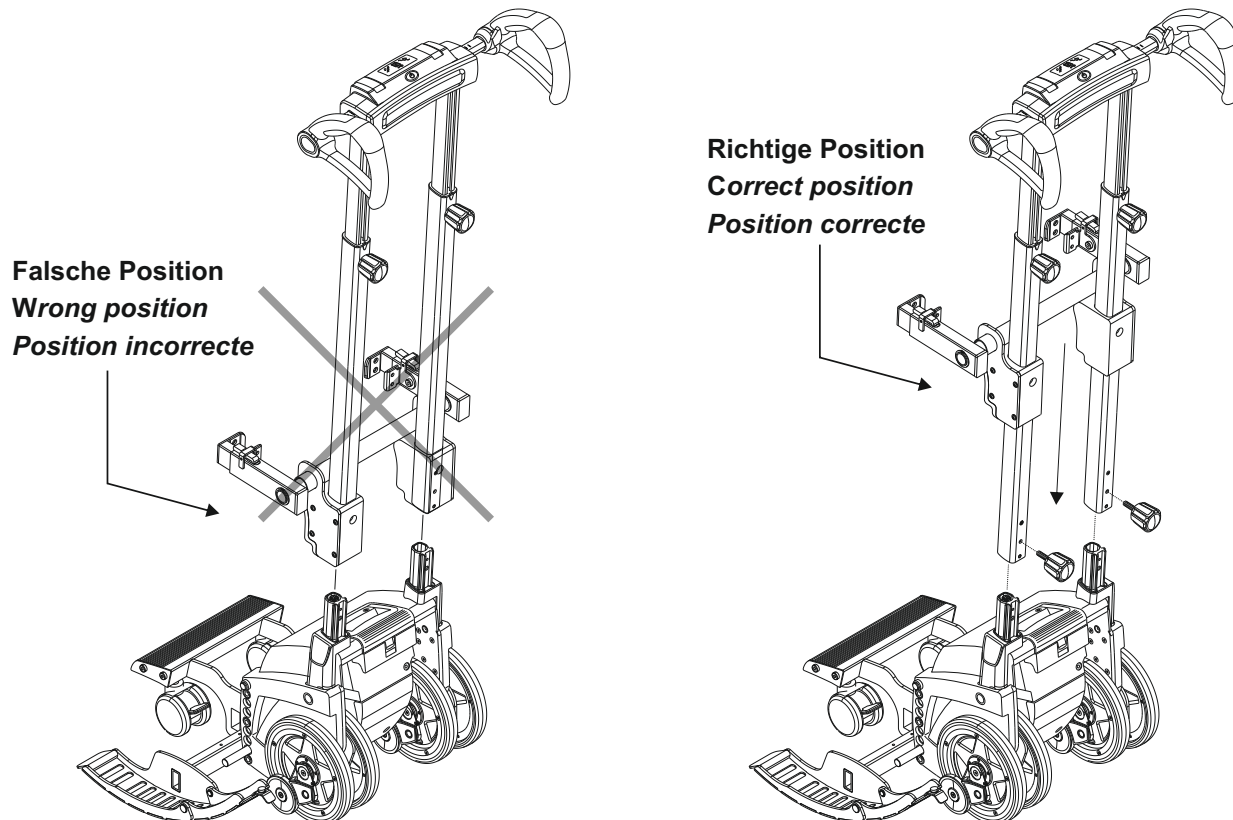
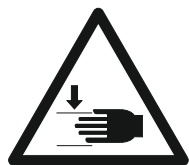
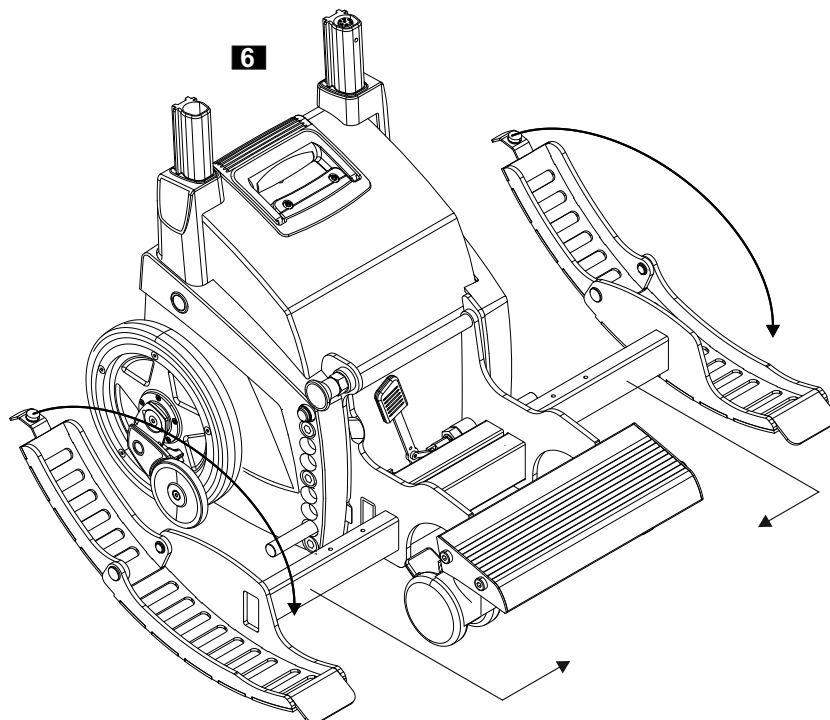
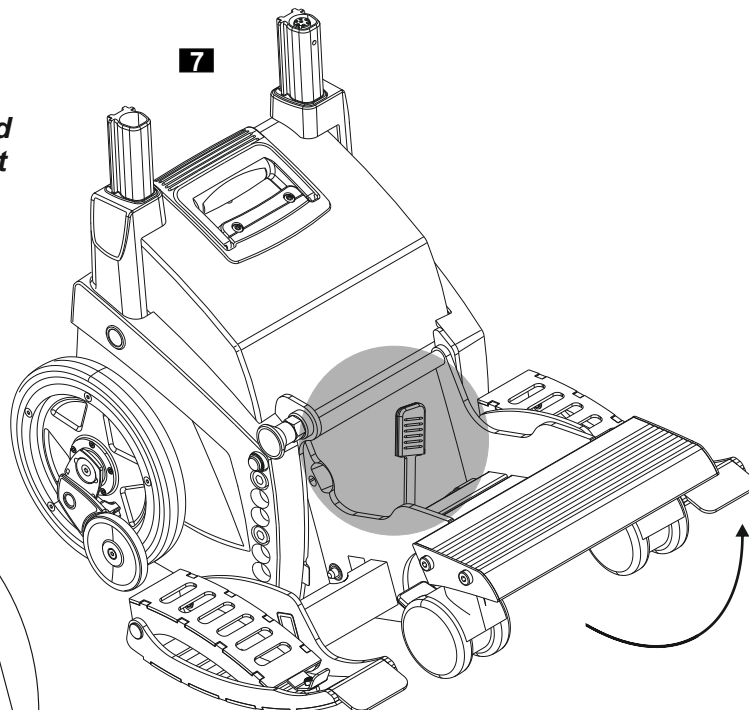
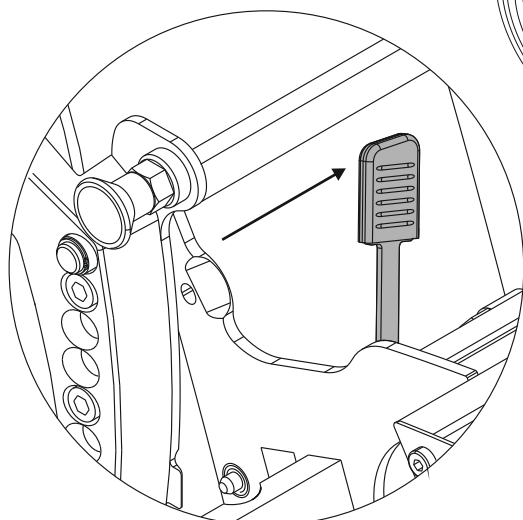
Transport
Transportation
Transport

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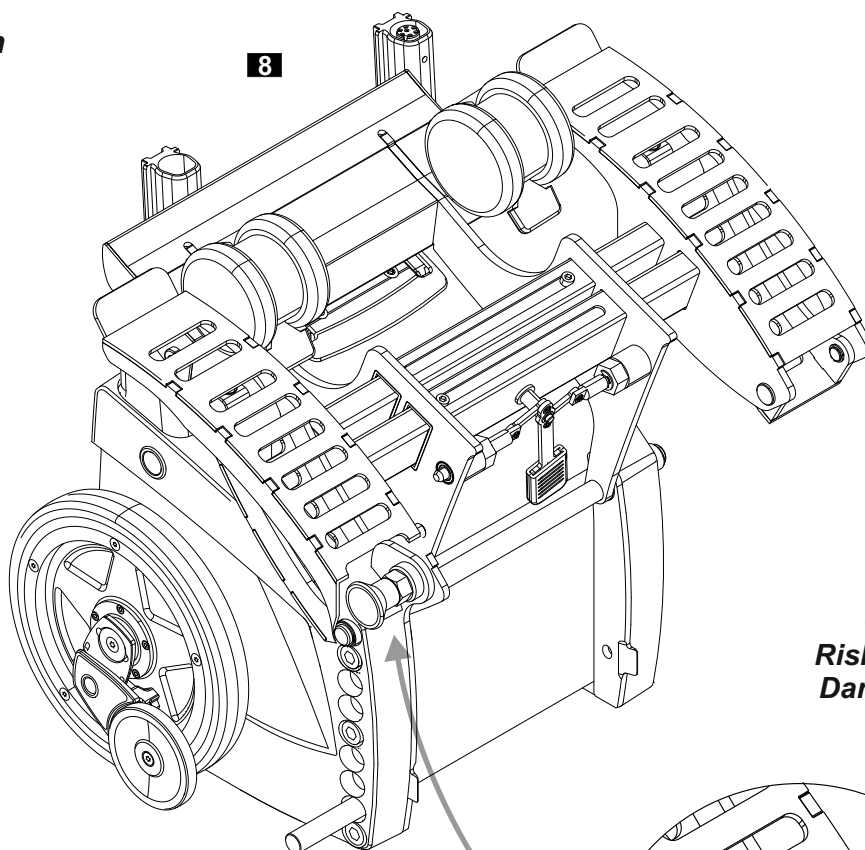
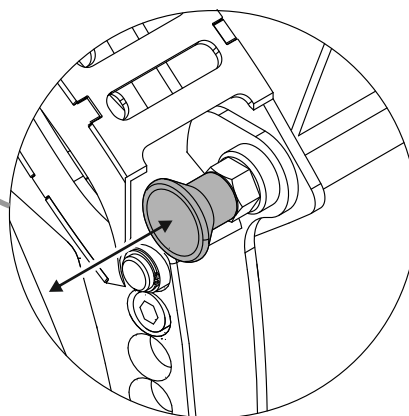
*For fixing respectively loosening of the lower wheelreceiver:
Press respectively pull out the locking pin*

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Ensure that the wheel holders are fixed before commissioning!

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ATTENTION

At the following pages the singel steps for mounting the sacalport X7 at the scalamobil are illustrated.

Any mounting must be done by personnel authorized by alber, not by the user of the scalamobil.

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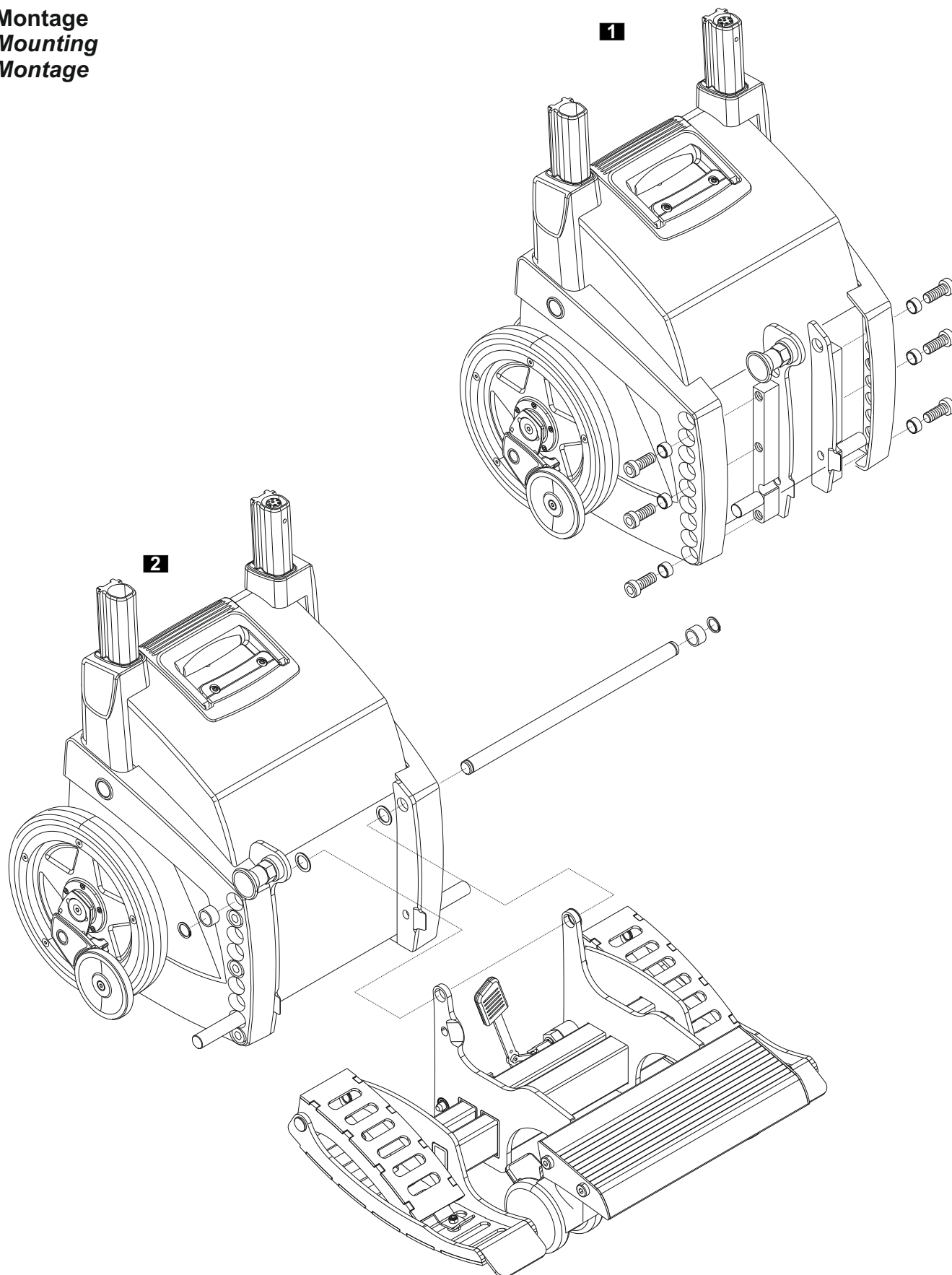
Montageanweisung X7 Instruction for mounting X 7 Instructions de montage X 7

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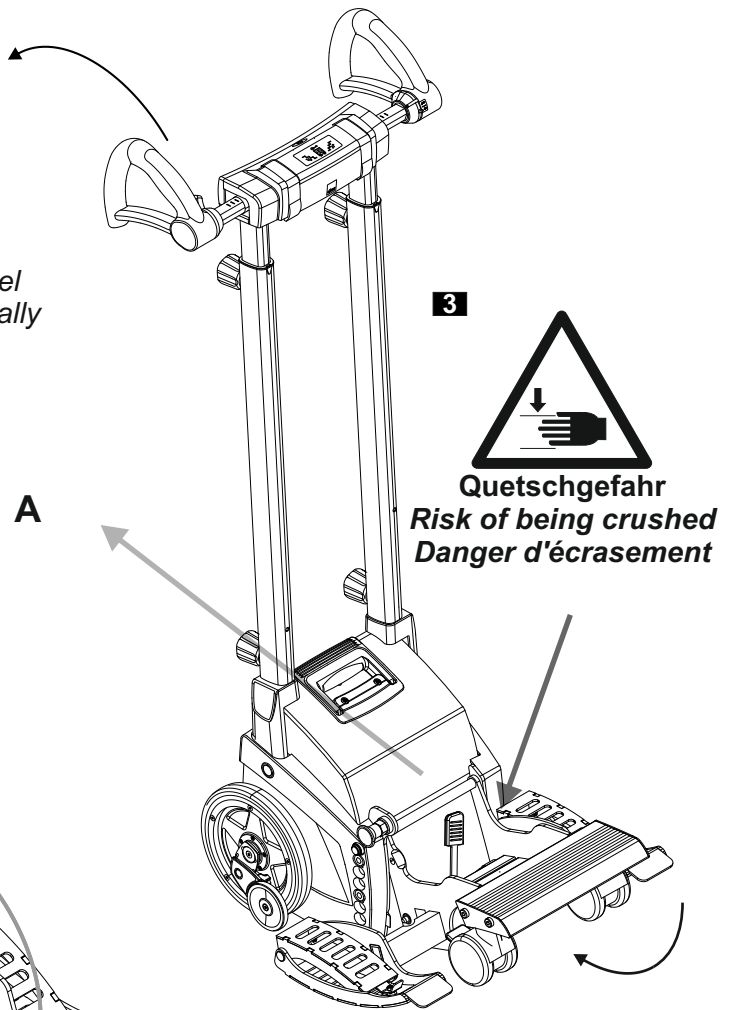
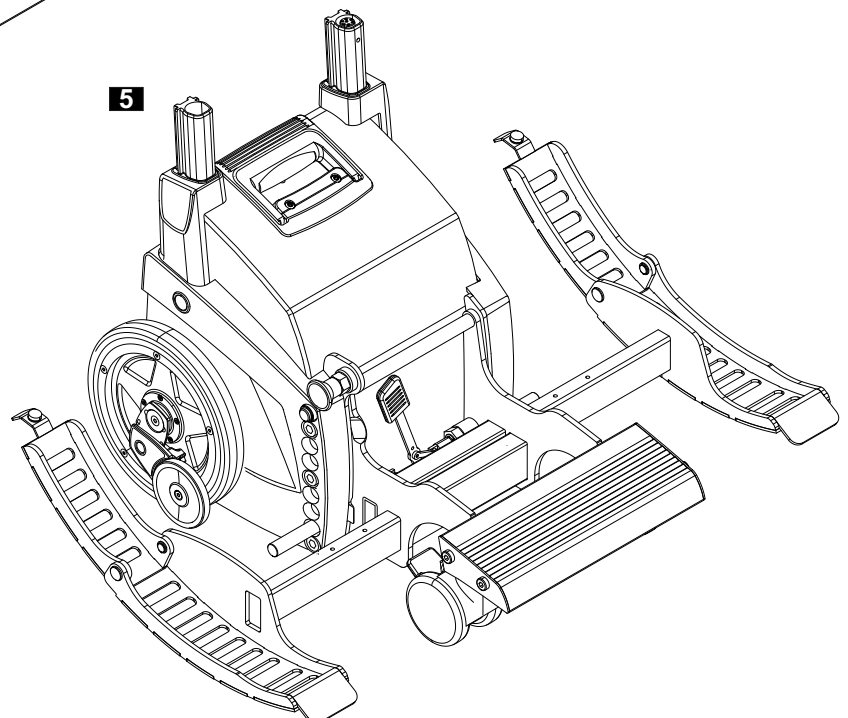
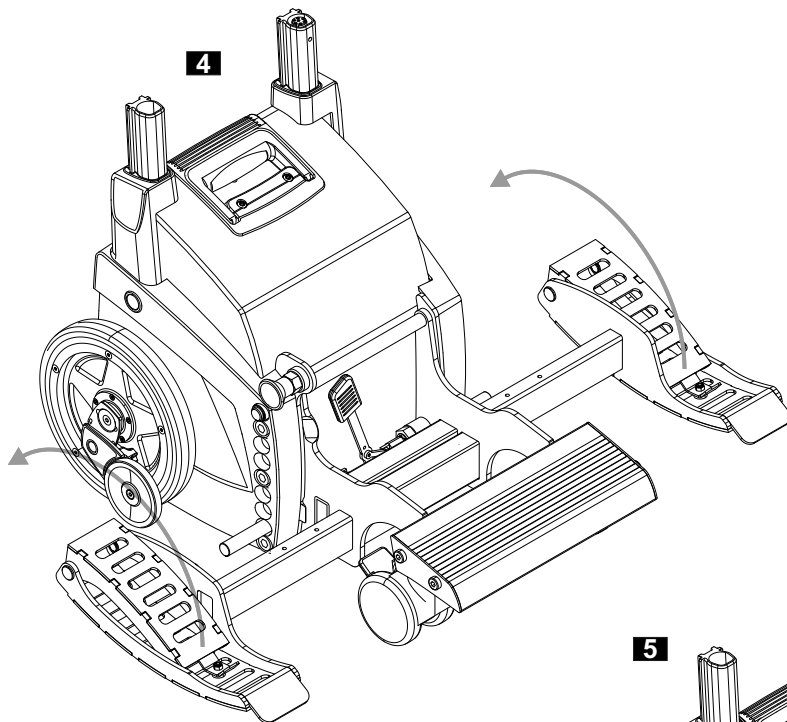
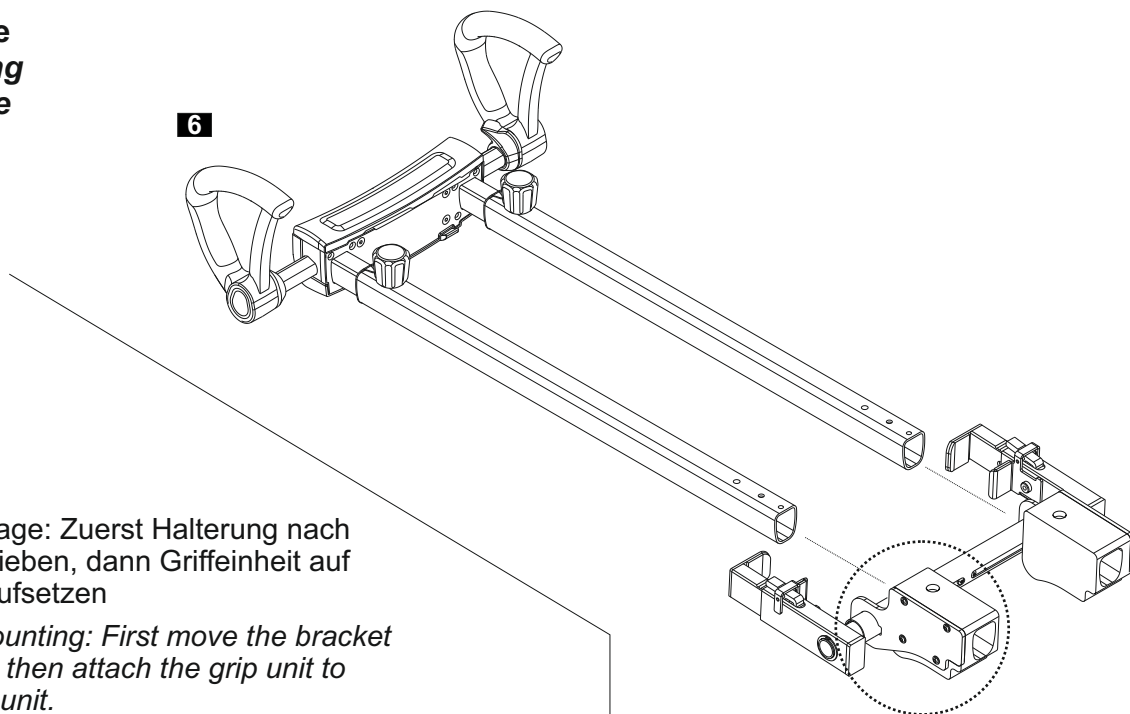
Tilt the scalamobil lowly backwards to fix the wheel receiver. If necessary: Push the lever [A] additionally to the right side.

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When mounting: First move the bracket upwards, then attach the grip unit to the drive unit.

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Alber GmbH