# Aquatec® Ocean / Ocean VIP / Ocean Dual-VIP / Ocean E-VIP

- EN** Calf- and amputee support / Calfsupport with foot support  
Assembly instruction
- DE** Waden- und Amputationsstütze / Wadenstütze mit Fußauflage  
Montageanweisung
- FR** Repose-mollet et repose-jambe pour amputé /  
Repose mollet avec repose-pied  
Instruction de montage

- IT** Poggiagambe per amputati e poggiapolpacci /  
poggiapolpacci con poggiapiedi  
Istruzioni di montaggio
- ES** Reposapierna para extremidad amputada y pantorrilla /  
Reposapantorrilla con reposapié  
Instrucciones de montaje
- NL** Kuit- en stompsteun / kuitsteun met voetsteun  
Montageaanwijzing
- DA** Benstøtte til læg og amputerede underben /  
lægstøtte med fodstøtte  
Monteringsanvisning
- FI** Pohje- ja amputoidun henkilön jalkatuki /  
pohjetuki, jossa jalkanoja  
Asennusohje
- NO** Støtte for legg og amputert ben / leggstøtte med fotstøtte  
Monteringsveiledning
- SV** Vad- och amputationsstöd / vadstöd med fotstöd  
Monteringsanvisning
- PT** Apoio de pernas para amputados e barriga das pernas /  
Apoio da barriga das pernas com apoio de pés  
Instruções de montagem
- CS** Opěrka lýtky a nohy osoby po amputaci /  
Opěrka lýtky s podnožkou  
Pokyny k sestavení
- LV** Apakšstilba un amputētas kājas balsts /  
apakšstilbu balsts ar pēdu balstu  
Uzstādīšanas norādījumi
- PL** Podparcie pod łydkę lub kikut nogi po amputacji /  
Podparcie pod łydkę z podnóżkiem  
Instrukcja montażu
- ET** Sääre- ja amputeeritud jala tugi / sääretugi jalatoega  
Koostejuhend

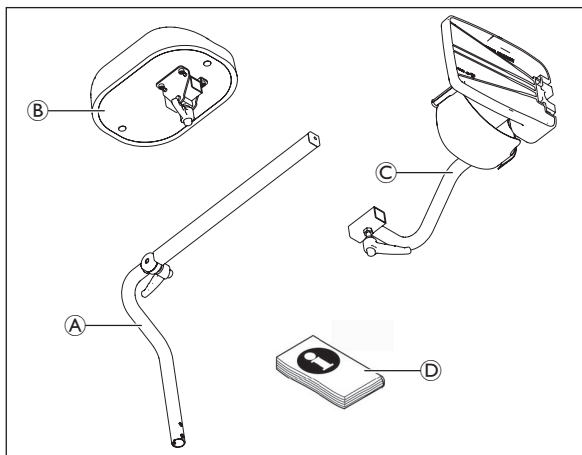
Fig. 1

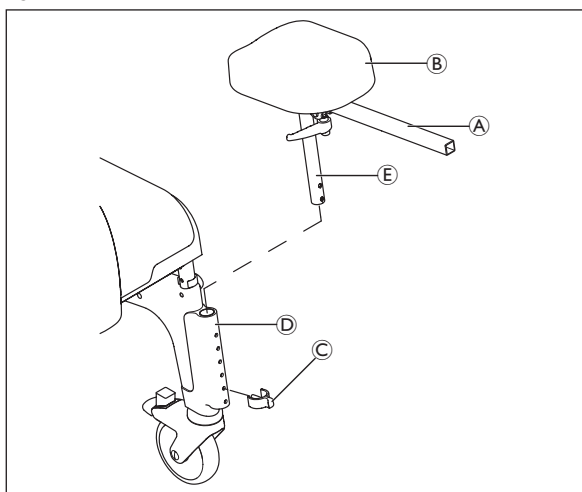
Fig. 2

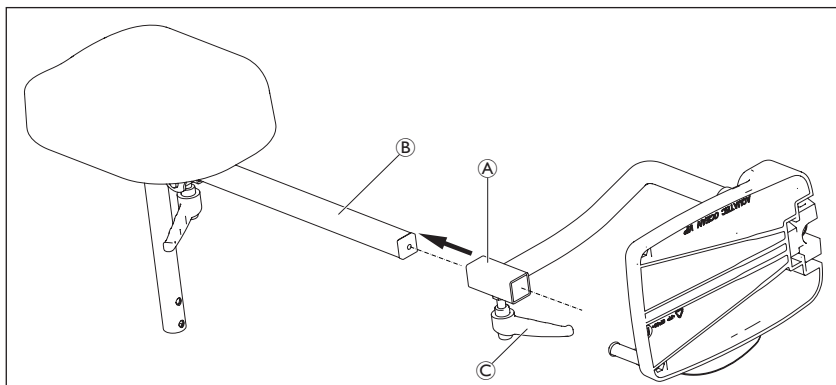
Fig. 3

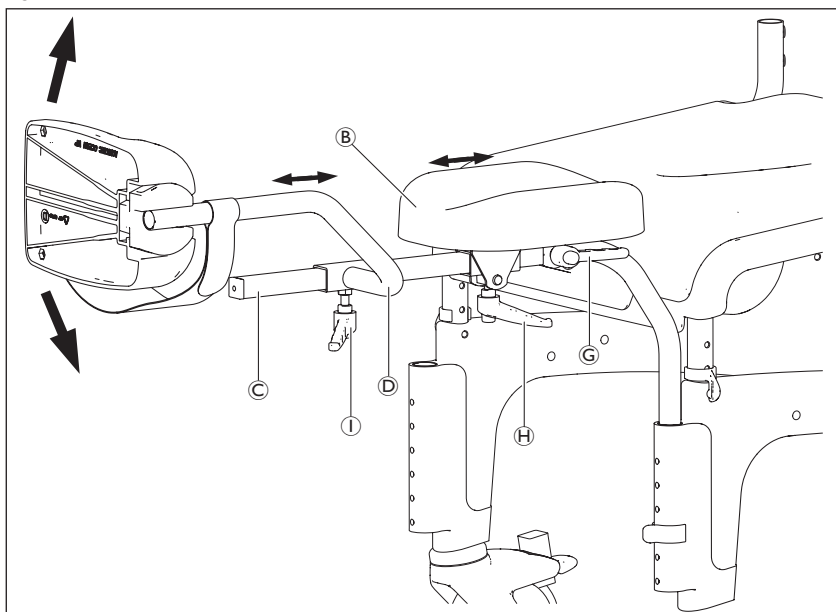
Fig. 4

This manual must be given to the user of the product.  
Before using the product, read this manual and save it  
for future reference!

**AQUATEC®**

Yes, you can.®

**Intended use**

The calf- and amputee leg rests are intended solely for use in conjunction with the Aquatec® Ocean range to ensure secure leg positioning. Any other use is prohibited.

**Warranty / Service Life**

See the user manual for the Ocean shower chair.

**Scope of delivery (Fig. 1)**

The following items are included in the scope of delivery:

1. Calf- and amputee leg rest with (A) and (B).
2. Calf rest and footrest with (A), (B) and (C).

(A)	Removable axle with adjusting rail
(B)	Support pad
(C)	Footrest with heel strap
(D)	Assembly instruction

**Safety****WARNING** Risk of injury!

- ▶ Do not use if defective.

**CAUTION**

- ▶ Check product regularly for damage and to ensure that it is securely assembled.
- ▶ Tighten screws and regularly check that they are secure.
- ▶ Be careful during assembly.
- ▶ Before use, adjust the calf- and amputee leg rest to suit the user.

**IMPORTANT** Risk of damage!

- ▶ Do not lift the product by the calf- and amputee leg rests.

**Fitting the calf- and amputee leg rest**

▶ Fig. 2

1. Push the support pad (B) onto the adjusting rail (A) and fix using the plastic lever.
2. Remove the clip (C) and remove the footrest from the shower chair.
3. Insert the removable axle (E) of the completely assembled calf- and amputee leg rest, into the footrest tube (D) of the shower chair, with the bend towards the inside, and set to the desired height using the previously removed clip (C).

**IMPORTANT**

- ▶ Ensure that the clip is seated properly in the borehole of the removable axle.

**Assembling the footrest**

▶ Fig. 3

1. Push the footrest with guide (A) onto the adjusting rail (B) and tighten the plastic lever (C) to the desired position.

The footrest can be mounted in front of or behind the support pad, depending on the required setting, in order to adjust the distance to the seat plate.

**Adjusting the support pad**

▶ Fig. 4

1. Release the plastic lever (H) of the support pad (B).
2. Move the support pad on the adjusting rail (C) into the desired position.
3. Retighten the plastic lever of the support pad.

The support pad can be turned through 180° when pushed onto the adjusting rail, depending on the required setting, in order to adjust the distance to the seat plate.

**Adjusting the length of the footrest**

▶ Fig. 4

1. Release the plastic lever (I) of the footrest.
2. Move the support pad with guide (D) on the adjusting rail (C) into the desired position.
3. Retighten the plastic lever (I).

**Setting the angle of the calf- and amputee leg rest**

▶ Fig. 4

1. Release the clamping lever (G) of the removable axle so that the embossed disks no longer interlock.
2. Swivel the complete calf- and amputee leg rest to the desired angle.
3. Re-tighten the clamping lever.

**IMPORTANT**

- ▶ When tightening the clamping lever, ensure that the embossed disks are properly interlocked.

**Cleaning**

Please note the cleaning instructions in the user manual for the Ocean shower chair.

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1. Løsne klemspaken © til den avtakbare akselen, slik at de pregede skivene ikke griper inn i hverandre.
Algol-Trehab Oy
AQUATEC Operations GmbH