# USER MANUAL

**VICAIR<sup>®</sup>**

**ACTIVE 02**

relief by air

<b>EN</b>	USER MANUAL	3
<b>NL</b>	GEBRUIKERSHANDLEIDING	9
<b>DE</b>	GEBRAUCHSANWEISUNG	15
<b>FR</b>	MANUEL DE L'UTILISATEUR	22
<b>DA</b>	BRUGERVEJLEDNING	29
<b>SV</b>	BRUKSANVISNING	35
<b>NO</b>	BRUKERMANUAL	41
<b>FI</b>	KÄYTTÖOHJE	47
<b>ES</b>	MANUAL DEL USUARIO	53
<b>PT</b>	MANUAL DE UTILIZADOR	59
<b>IT</b>	MANUALE UTENTE	65
<b>GR</b>	ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ	71
<b>HR</b>	KORISNIČKI PRIRUČNIK	78
<b>SL</b>	UPORABNIŠKI PRIROČNIK	84
<b>PL</b>	INSTRUKCJA OBSŁUGI	90
<b>ZH</b>	用户手册	96
<b>JP</b>	ユーザーマニュアル	101

Dear customer,

Thank you for choosing a Vicair® product. We are sure you will appreciate the simplicity, reliability and fantastic effect. For quick installation you can use the quick installation guide provided with your cushion. In addition, please read and understand this user manual before use. Save both the Quick Installation Guide and this manual for future reference.

For more information or to (re)order manuals, contact your country's distributor.


Go to [vicair.com](http://vicair.com) for distributor information.

## INTRODUCTION

The Vicair Active O2 is a wheelchair seat cushion that provides a high level of skin protection for active wheelchair users with a high risk of developing pressure ulcers. The Vicair Active O2 allows easy transfers and provides stable body support, comfort and pressure ulcer prevention. It is also a high performance addition to a pressure ulcer treatment plan (stage I-IV).

The Vicair Active O2 cushion is machine washable without removing the SmartCells! You only have to take out the foam inlay.

## MAIN COMPONENTS

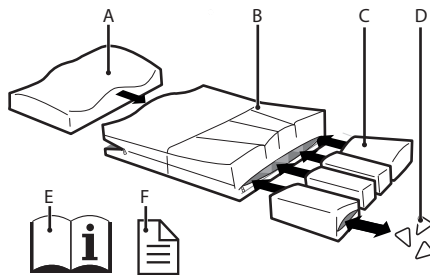- A** Foam inlay with fluid-resistant foam inlay liner, placed in front compartment
- B** Vicair Active O2 Cover
- C** SmartCell liners, placed in four rear compartments
- D** SmartCells™
- E** User manual
- F** Quick Installation Guide

## GENERAL PRODUCT SPECIFICATIONS

Product name:	Vicair Active O2
Manufactured by:	Vicair B.V., The Netherlands
Warranty:	24 months
Weight (excl. cover):	Smallest size:
	1 kg / 2.2 lbs
	Medium size:
	1,2 kg / 2.6 lbs
	Largest size:
	1,5 kg / 3.3 lbs
Available sizes:	See the grid in the back of the manual
Used materials:	All latex free
SmartCell liners:	Polyurethane coated polyester fabric
SmartCell liner filling:	Air-filled SmartCells with low friction coating

Foam inlay:	Polyether
Foam inlay liner:	Fluid-resistant polyurethane coated polyester fabric
Active O2 Cover:	Top: Knitted polyester mesh fabric. Sides: Polyurethane coated nylon fabric. Bottom: Knitted polyester mesh fabric.

Covers are machine washable. Covers comply with EN1021-1, 1021-2 and CAL117.

## PRECAUTIONS AND SAFETY INSTRUCTIONS

---

A Vicair wheelchair seat cushion may only be used as a wheelchair seat cushion. The use of this product in any other way, is considered to be misuse. Vicair B.V. cannot be held responsible for damage caused by misuse, improper use or negligence. Make sure that you read and understand this manual completely.

**Warning!** Vicair B.V. recommends to install the product under direct supervision of a therapist or technical advisor.

**Warning!** Your tailbone and pelvic bones should not touch the bottom of the cushion. Checking for “bottoming out” should be done during first use and after adjustments to the cushion filling.

**Warning!** Make sure that your tailbone and pelvic bones are resting on the rear compartments with SmartCell liners, not on the foam inlay.

**Warning!** Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

**Warning!** Make sure the SmartCell liners are placed in the correct compartment (see SmartCell liner label).

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

**Warning!** No testing data is currently available about the safe use of Vicair wheelchair cushions in motor vehicles. Therefore we do not encourage using the wheelchair cushion in motor vehicles.

**Warning!** In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination.

**Warning!** Every three months, check the product for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the product. Consult your dealer.

**Caution!** Do not use or leave the product close to a naked flame or a high temperature heat source.

**Caution!** Do not steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product will age more rapid.

**Caution!** Remove foam inlay before washing this cushion in a washing machine.

**Caution!** Do not use the foam inlay without the foam inlay liner.

**Caution!** Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

**Caution!** Do not place the topside of the Vicair O2 cushion against the glass of the washer door.

**Caution!** Make sure the cushion cannot interfere with the wheel spokes.

## INSTALLATION

---

**Warning!** Vicair B.V. recommends to install the product under direct supervision of a therapist or technical advisor.

**Warning!** Make sure that your tailbone and pelvic bones are resting on the rear

compartments with SmartCell liners, not on the foam inlay.

**Caution!** Make sure the cushion cannot interfere with the wheel spokes.

**Caution!** Do not use the foam inlay without the foam inlay liner.

**Notice!** Make sure that the wheelchair seat is clean and degreased.

**Tip:** When you prefer not to use the Velcro, you can remove the Velcro strips. Note that the strips confected to the bottom of the cushion cannot be removed.

### **Perform the following steps when installing the Vicair Active O2**

These steps correspond to the installation images 1-5 on the Quick Installation Guide. Image i (I, II & III) shows how to (re)assemble the cushion (chapter 'Maintenance and cleaning'):

- Check cushion size. Width (X) and depth (Y) of the cushion and seat should match.
- Remove protective backing tape from self-adhesive Velcro strips attached to the bottom of the cushion.
- Place cushion by pressing firmly. Make sure the foam inlay is at the front of the wheelchair seat.
- Sit on the cushion for a few minutes, to allow the cushion to adapt to your body shape.
- Sit on the cushion for a few minutes, to allow the cushion to adapt to your body shape. Make sure that your tailbone and pelvic bones are resting on the rear compartments with SmartCell liners, not on the foam inlay.
- If necessary, adjust footrest(s).

### **Checking for "bottoming out":**

Image 6 - Quick Installation Guide

**Warning!** Your tailbone and pelvic bones should not touch the bottom of the cushion. Checking for "bottoming out" should be done during first use and after adjustments to the cushion filling.

**Warning!** Make sure that your tailbone and pelvic bones are resting on the rear compartments with SmartCell liners, not on the foam inlay.

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

- Check for bottoming out, after sitting on the cushion for approximately 5 minutes.
- Let your therapist or technical advisor slide a hand between your buttocks and the Vicair cushion while leaning forward.
- Ask your therapist or technical advisor to check if your lowest bony prominences are supported by at least 2,5 cm / 1" of SmartCells™. This prevents your seat bones from touching the wheelchair seat.
- Do not use the cushion if bottoming out occurs. In most cases, adjusting the filling grade of the SmartCell liners will solve the problem.

## **USE**

---

### **Checking for redness of the skin**

Image 7 - Quick Installation Guide

**Warning!** Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

### **Adjusting the filling grade**

**Warning!** Vicair recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

### **Ready for use**

This Vicair product is ready for use. It has its own unique filling grade, depending on size and model. However – if necessary – the filling grade can be adjusted by removing or adding SmartCells from/to the various SmartCell liners.

You can find the default filling grade on the label inside the cover.

**Warning!** Make sure the SmartCell liners are placed in the correct compartment (see SmartCell liner label).

### Conditions for use and storage

**Caution!** Do not use or leave the product close to a naked flame or a high temperature heat source.

- **User weight:**

Minimum: 15 kg / 33 lbs,  
maximum: 200 kg / 441 lbs

- **Temperature:**

Minimum: -20 °C / 4 °F,  
maximum: 55 °C / 131 °F

- **Use at high altitude**

**Notice!** Vicair B.V. recommends not to use the cushion above 2400 m (8000 ft) above sea level or below 50 m (164 ft) below sea level. Because each SmartCell is filled with air, the cushion may behave differently when used at high altitude or in an airplane. Under such circumstances, the cushion's firmness can be adjusted by removing a few cells from each SmartCell liner.

- **Air humidity:**

Minimum: 0%, maximum 100%

### Daily fluff-up

Vicair B.V. recommends to fluff-up the four rear compartments of your Vicair Active O2 cushion before use on a daily basis.

### Use of Vicair seat cushions in motor vehicles

**Warning!** No testing data is currently available about the safe use of Vicair wheelchair cushions in motor vehicles. Therefore we do not encourage using the wheelchair cushion in motor vehicles.

As an alternative for your Vicair wheelchair cushion, the Vicair AllRounder O2 (which can be

fastened to the user's body) can be used if you require additional protection during transportation.

### Re-use - Transferring the product to another user

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. The product has to be thoroughly cleaned and disinfected, before another person is allowed to use the it. See chapter 'Maintenance and cleaning' for instructions.

**Notice!** Make sure the product is correctly adjusted to the other person.

### Replacing the cover

Under normal use, Vicair B.V. recommends to replace the cover every year. Replacement covers can be purchased separately.

### Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulation. Ensure the product is cleaned prior to disposal to avoid any risk of contamination.

## MAINTENANCE AND CLEANING

---

**Warning!** Every three months, check the product for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the product. Consult your dealer.

**Warning!** Make sure the SmartCell liners are placed in the correct compartment (see SmartCell liner label).

**Caution!** Do not steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product will age more rapid.

**Caution!** Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

**Caution!** Remove foam inlay before washing this cushion in a washing machine.

**Caution!** Do not use the foam inlay without the foam inlay liner.

**Caution!** Do not place the topside of the Vicair O2 cushion against the glass of the washer door.

### Cleaning the Vicair Active O2

After you have emptied the storage pouch and removed the foam inlay, your Vicair Active O2 cushion can be machine washed at 60°C / 140 °F using an anti-crease cycle and mild detergent. You do not need to remove the SmartCell liners before putting the cushion into the washing machine. Make sure the topside of the cushion is not placed against the glass of the washer door. Do not load the machine with more than half of its maximum load weight. Spin speed should be reduced. When using a machine with agitator, make sure that the cushion can move freely within the drum. Do not use any chlorine bleach and do not dry clean! Do not tumble dry your Vicair Active O2 cushion but let it air dry (5-6 hours).

### Cleaning the Vicair Active O2 foam inlay

Do not machine wash the foam inlay and do not immerse it in water or any other liquid. To clean the foam inlay you first remove it from its compartment.

- **Cleaning the foam inlay liner**

Take the foam out of the foam inlay liner. The emptied liner can be machine washed at 60°C. Follow the washing instructions on the label attached to the inside of the liner.

- **Cleaning the foam**

Take the foam out of the foam inlay liner and clean it by using a disinfectant spray.

### Disinfecting your Vicair Active O2

**Warning!** In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination. Clean the Active O2 cover, foam inlay and SmartCell liners as instructed above. Preferably use an anti-bacterial detergent. After cleaning spray your cover, foam inlay liner and SmartCell liners with a disinfectant. Bend the cover compartments outwards to make sure the spray gets into the gaps.

### (Re)assembling your Vicair Active O2

The Vicair Active O2 is fully assembled on delivery. When it needs to be (re)assembled, for instance after cleaning, follow the assembly instructions underneath. The instructions correspond to image i (I, II & III) on the Quick Installation Guide.

### Placing the foam inlay in the Vicair Active O2 cover (image i - I)

**Caution!** Do not use the foam inlay without the foam inlay liner.

- Completely open the zipper at the front of the Active O2 cover.
- Place the foam inlay in the Active O2 cover, the sloping side pointing to the back of the cushion and the contoured side on top.
- Close the zipper. Make sure you do not damage the foam inlay liner.

### Adjusting the number of SmartCells in the SmartCell liners (image i - II)

**Warning!** Vicair B.V. recommends to adjust the filling grade under the direct supervision of your therapist or technical advisor.

The number of SmartCells in each SmartCell liner depends on the size of the cushion. You can find the default filling grade on the label inside the cushion cover. When it's necessary to adjust the number of SmartCells, follow these steps:

- Open the SmartCell liner's zipper and add or remove the desired number of SmartCells.
- Close the zipper. Make sure that you do not damage the SmartCells.

### **Placing the SmartCell liners in the Vicair Active O2 cover (image i - III)**

**Warning!** Make sure the SmartCell liners are placed in the correct compartment (see SmartCell liner label).

- Completely open the zipper at the back of the cushion cover.
- Place the SmartCell liners in the cover. Make sure you place the correct SmartCell liner in each compartment. The SmartCell liner's label states where it should be placed.
- Close the zipper. Make sure that you do not damage the SmartCell liners.

## **WARRANTY**

---

Please see [www.vicair.com](http://www.vicair.com) for complete information on warranty processes, terms and conditions.

Vicair B.V. offers a limited warranty against defects in workmanship and materials, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning, or negligence, is excluded from this warranty.

### **Warranty period**

From original date of purchase: 24 months.  
The SmartCells in this Vicair O2 cushion have a lifelong warranty period.

Should a defect in materials or workmanship occur within the above mentioned warranty period, Vicair B.V. will – at its own choice – repair or replace the product free of charge. Remedies for breach of express warranties herein are limited to repair or replacement of the product.

In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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